Before you begin

- Please read the instructions for use in its entirety, prior to use.
- Cover your work space with a clean, disposable absorbent workspace cover.
- All components and specimen must be brought to room temperature (between 15-30°C) prior to testing.
- Do not use the test more than once.

Preparation

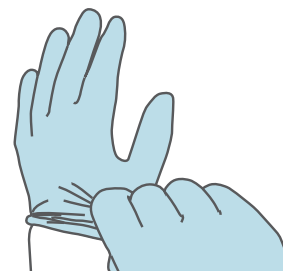
- Test pouch (test device)
- Assay diluent (for Dengue IgG/IgM)
- Disposable capillary pipette (10 µl) (for Dengue IgG/IgM)
- Disposable dropper (for Dengue NS1 Ag)
- Timer & disposable gloves (not provided)

For more detailed instructions please refer to Instructions for use.



1 Check the expiration date.
If the device is expired, use another kit.


2 Put on disposable gloves.
Use new gloves for each patient.

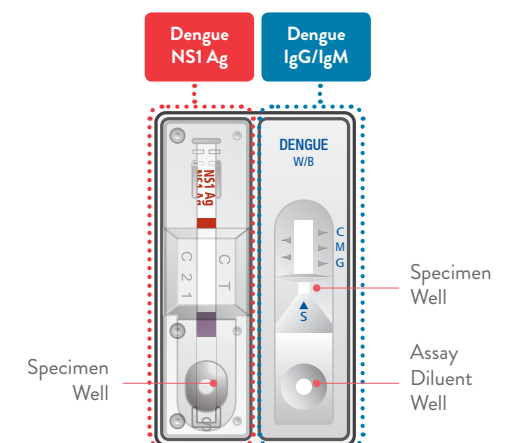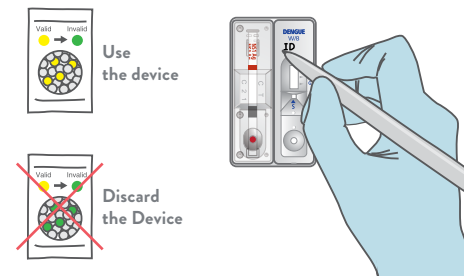
3 Open the test pouch and check the desiccant. **If the color is green, use another kit.** Write the patient's name on the test device.

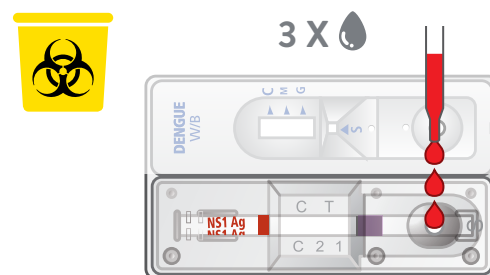
4 DENGUE NS1 Ag
Add **3 drops (100 µl)** of specimen into the specimen well.
Discard the dropper in the biohazard box after use.

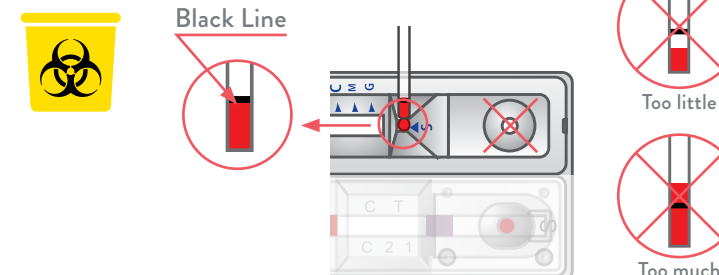5 DENGUE IgG/IgM
Draw specimen to **black line (10 µl)** then add into the square specimen well "S".
Discard the pipette in the biohazard box after use.

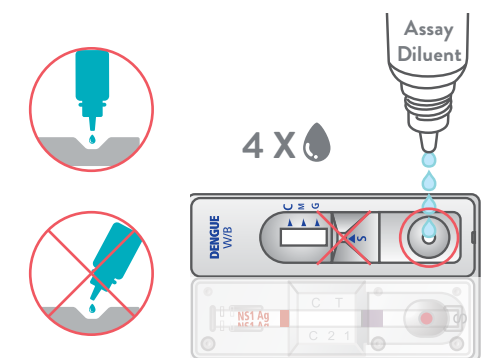6 DENGUE IgG/IgM
Add **4 drops** of assay diluent into the round assay diluent well.

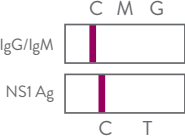
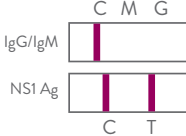
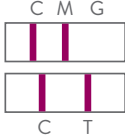
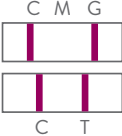
7 Read results between **15-20 minutes** after adding diluent.

Do not read after 20 minutes.
Reading outside of this time frame may provide false results.

Caution: Visible test lines should be considered positive, even if faint.

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