

Developed and Manufactured by:



Distributed by:



Item

Unit Description No.

Lactoferrin Demonstration Panel

ED355

INTENDED USE

FOR DEMONSTRATION PURPOSES ONLY – for *in vitro* use only

Intended for use only with the **LEUKO EZ VUE®** assay. This panel should not be used as a replacement for kit controls or for the reporting of patient results.

EXPLANATION

The Lactoferrin Demonstration Panel consists of representative reagents of various lactoferrin concentrations that can be used to demonstrate performance of the test listed above. Also included are negative reagents that do not contain lactoferrin.

MATERIALS PROVIDED

ED355A	Lactoferrin (-) Reagent, 800 µL
ED355B	Lactoferrin (+) Reagent, 800 µL
ED355C	Lactoferrin (+) Reagent, 800 µL
ED355D	Lactoferrin (+) Reagent, 800 µL
ED355E	Lactoferrin (+) Reagent, 800 µL
ED355F	Lactoferrin (+) Reagent, 800 µL
ED355G	Lactoferrin (-) Reagent, 800 µL
ED355H	Lactoferrin (+) Reagent, 800 µL
ED355I	Lactoferrin (-) Reagent, 800 µL
ED355J	Lactoferrin (+) Reagent, 800 µL

PRECAUTIONS

- Reagents should be stored between 2° and 8°C and should be kept tightly closed when not in use.
- Reagents should be at room temperature before use.
- The reagents contain lactoferrin, which is a human derived material. Material has been tested and found negative for antibody to HIV-1, HIV-2, HCV, and HbsAg. No known test method can offer complete assurance that infectious agents are absent. **All human source products should be handled as potentially infectious material.** A procedure for handling biohazards is published in the CDC/NIH *Manual of Biosafety in Microbiology & Biomedical Laboratories*.

PROCEDURE

Follow the procedure for the **LEUKO EZ VUE®** assay for preparing each reagent as if it were a fecal specimen.

INTERPRETATION

Refer to the **LEUKO EZ VUE®** package insert for interpretation of results. Confirm observed results with the corresponding description listed under MATERIALS PROVIDED in this insert. If observed results do not match the description, please contact Alere Technical Service at 1-877-866-9335 or email TS.SCR@alere.com.

Developed and Manufactured by:



TECHLAB, Inc.
2001 Kraft Drive,
Blacksburg, VA 24060-6358 USA

Distributed by:



Alere North America, LLC
30 South Keller Road
Orlando, Florida 32810
TEL 1-877-441-7440

The Alere Logo and Alere are trademarks of the Alere group of companies.

LEUKO EZ VUE, the TECHLAB Logo, and TECHLAB are trademarks of TECHLAB, Inc., under license.

Made in the USA

© 2015 TECHLAB, Inc. All rights reserved.