



ID NOW[™] RSV

QUICK REFERENCE INSTRUCTIONS

Materials Required to Run a Test:



Test
Base



Sample
Receiver




Transfer
Cartridge



Nasopharyngeal
Swab

Instrument:

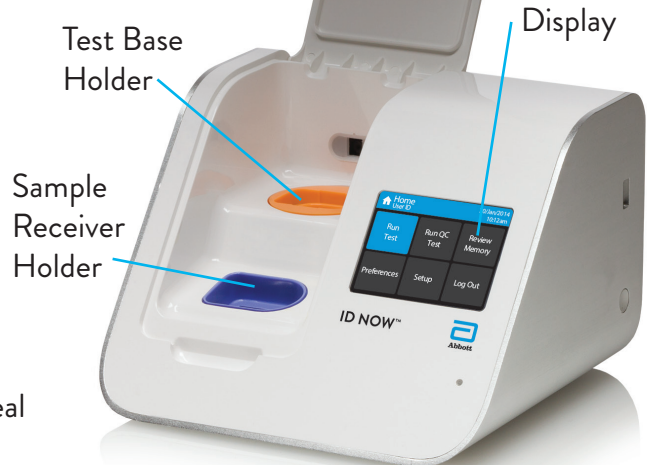

CLIA COMPLEXITY: WAIVED


Refer to the ID NOW RSV Product Insert and User Manual for complete instructions.

External Positive and a Sterile Swab for Negative Control should be tested once with each new shipment received and once for each untrained operator.
For QC testing, select Run QC Test on the Home screen, and follow the displayed instructions. Refer to Running a QC Test in the ID NOW Instrument User Manual for further details.

1 Touch 'Run QC Test'.



2 Select 'RSV' from the menu.

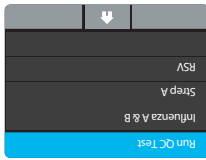


3 Select the QC Test to be Run.



4 Confirm Test

Confirm the test type to match the QC sample intended for testing by touching 'OK' and following the on screen prompts to complete testing.

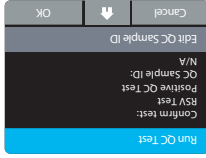

Note: The QC test is run in the same manner as a Direct Nasopharyngeal Swab Patient Test. See the other side for step by step instructions.



Abbott Diagnostics Scarborough, Inc.
10 Southgate Road
Scarborough, Maine 04074 USA
www.globalpointofcare.abbott

US
+1 855 731 2288
ts.scr@abbott.com
Africa, Russia, CIS
+44 161 483 9032
EMProductsupport@abbott.com
Asia Pacific
+61 7 3363 7711
APProductsupport@abbott.com
Canada
+1 800 818 8335
CANProductsupport@abbott.com
Europe & Middle East
+44 161 483 9032
EMProductsupport@abbott.com
Latin America
+57 (1) 4824033
LAPProductsupport@abbott.com

Technical Support Advice Line
Further information can be obtained from your distributor, or by contacting Technical Support on:

ID NOW™ RSV QUICK REFERENCE INSTRUCTIONS

Before performing this test, refer to the ID NOW RSV Product Insert and User Manual for complete Test Procedure and additional information.

Follow the step-by-step instructions shown on the instrument screen.

1



2

15°C → 30°C



3

15°C → 30°C



4




4a



4b







5

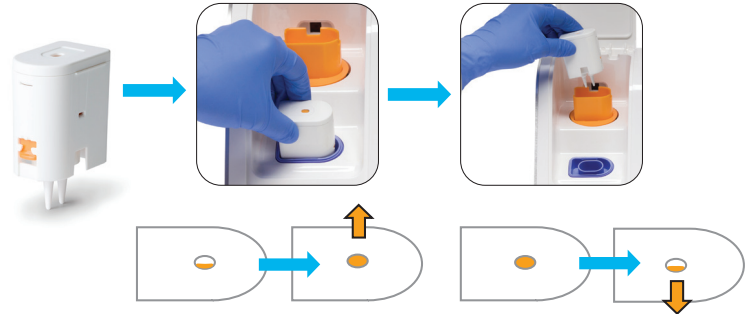

6




7

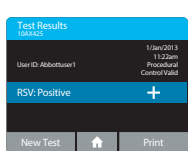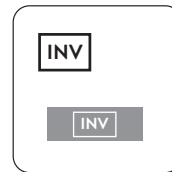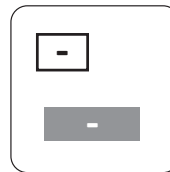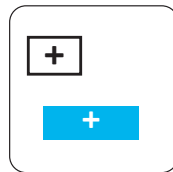



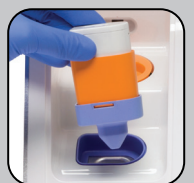

Refer to Product Insert instructions for retest procedure of an Invalid test result.

8



Refer to the Product Insert for instructions on safe handling and disposal of samples and test components.

Abbott ID NOW RSV QRI, EN Size: 8.5 in. x 11 in.	Printed Colors  CMYK	PN: IN435001 Rev: 6 Date of Last Revision: 6.2 2020/08/31
	Incoming Inspection Colors (For Reference Only) Colors below are not used for printing  PMS 2995 U Primary Blue  PMS 303 U Dark Blue  PMS 324 U Mint	