




**Abbott**

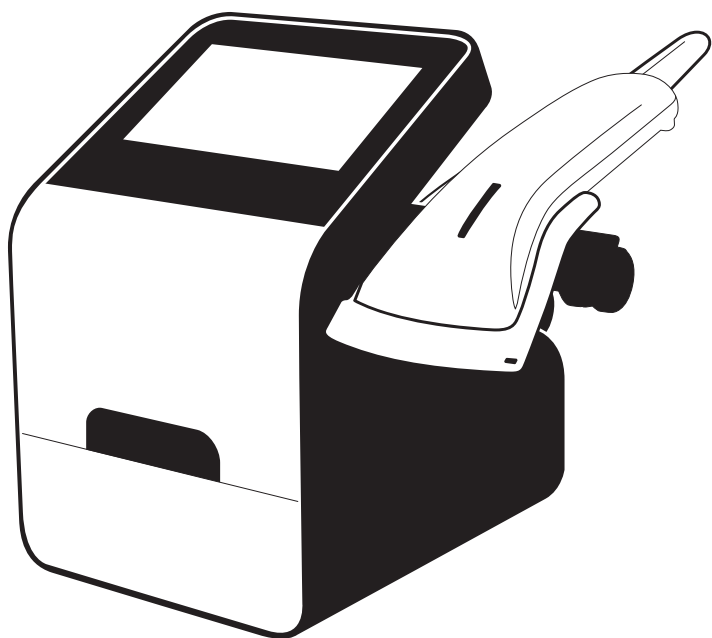

# **DIGIVAL™**

## QUICK START GUIDE



[www.globalpointofcare.eifu.abbott](http://www.globalpointofcare.eifu.abbott)

Contents

REF LFR-000

REF LFR-024



DIGIVAL™

REF L22XWU1-200



Barcode scanner and holster




Barcode scanner bracket



Power Supply

REF INLFR000

REF INLFR024

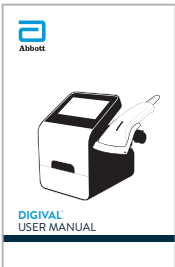

User Manual

REF LFR-003



Calibration Check Card Pack

# Contents

**REF**    LFR-004



DIGIVAL™ Tray 1 Pack


# Optional Accessories

**REF**    MCP1810395



Printer

**REF**    EQ004003

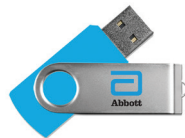

Abbott USB Memory Stick

*Unpack the shipping container and check for any obvious damage.*

## **DIGIVAL™ Set Up**


Place the DIGIVAL on a level bench top within access to a power supply.



- ❶ Barcode scanner bracket location holes
- ❷ USB port (x1)
- ❸ Power connection
- ❹ Ethernet
- ❺ USB ports (x2)




Assembly of barcode scanner holster.

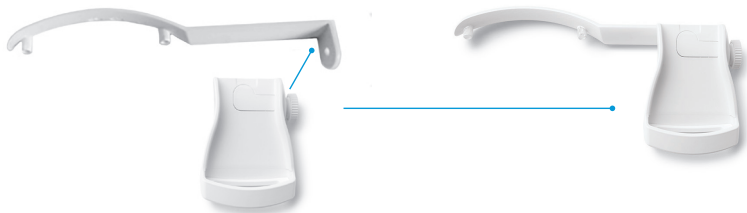


Connect bracket assembly to back of the DIGIVAL. Attach barcode scanner to USB and place onto holster.




Connect power supply and turn on the DIGIVAL.

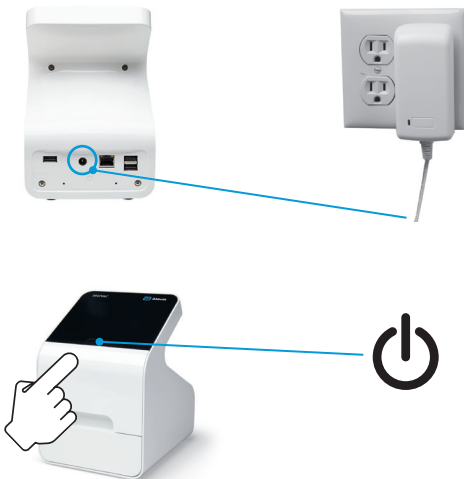


Open drawer, remove tray, and place calibration check card into position as shown on screen.

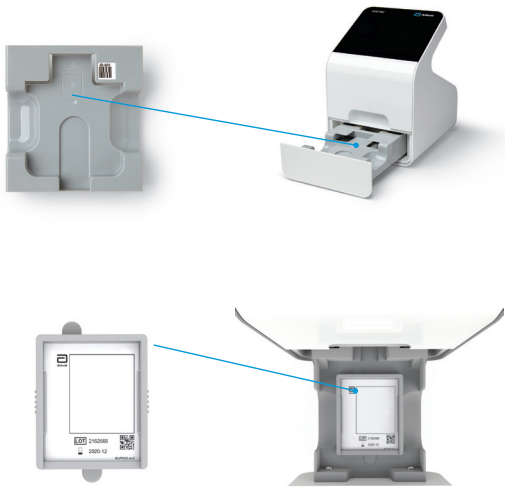


Enter the admin password  
Enter 'admin'

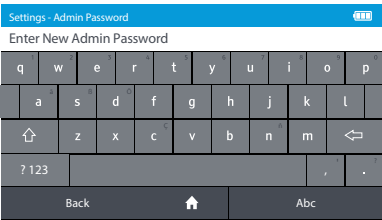


Set the Date

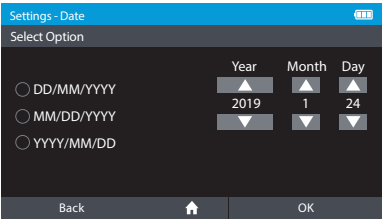

## Set the Time

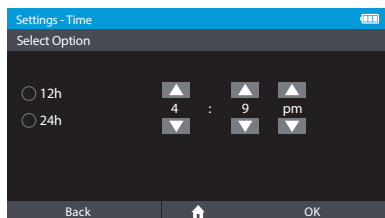

## Check Instrument Calibration

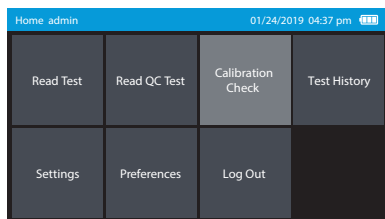

## Ordering and Contact Information

**REF** LFR-000: DIGIVAL - OUS

+1 321 441 7200

**REF** LFR-024: DIGIVAL - US

+1 877 441 7440

## Technical Support

### US

+1 888 735 5317

ts.scr@abbott.com

### Africa, Russia, CIS

+44 161 483 9032

EME.TechSupport@abbott.com

### Asia Pacific

+61 7 3363 7711

AP.TechSupport@abbott.com

### Canada

+1 800 818 8335

CANproductsupport@abbott.com

### Europe & Middle East

+44 161 483 9032


EME.TechSupport@abbott.com

### Latin America

+57 601 482 4033

LA.TechSupport@abbott.com



 **Abbott Diagnostics Scarborough, Inc.**  
10 Southgate Road  
Scarborough, Maine 04074 USA  
[www.globalpointofcare.abbott](http://www.globalpointofcare.abbott)



[www.globalpointofcare.eifu.abbott](http://www.globalpointofcare.eifu.abbott)






© 2024 Abbott. All rights reserved.

All trademarks referenced are trademarks of either the Abbott group of companies or their respective owners.

INLFR009 Rev. 7 2024/03  
SAP: 40005340



<div>Abbott DIGIVAL</div> <div>QSG, EN</div> <div>Size: 5.25 in x 8.5 in</div>	<div>Printed Colors</div> <div>CMYK</div>	<div>PN: INLFR009 Rev: 7 SAP: 40005340</div> <div>Date of Last Revision: 7.3 2024/03/11</div>
	<div>Incoming Inspection Colors (For Reference Only)</div> <div>Colors below are not used for printing</div> <div><div>PMS 2995 U Primary Blue</div><div>PMS 303 U Dark Blue</div></div>	