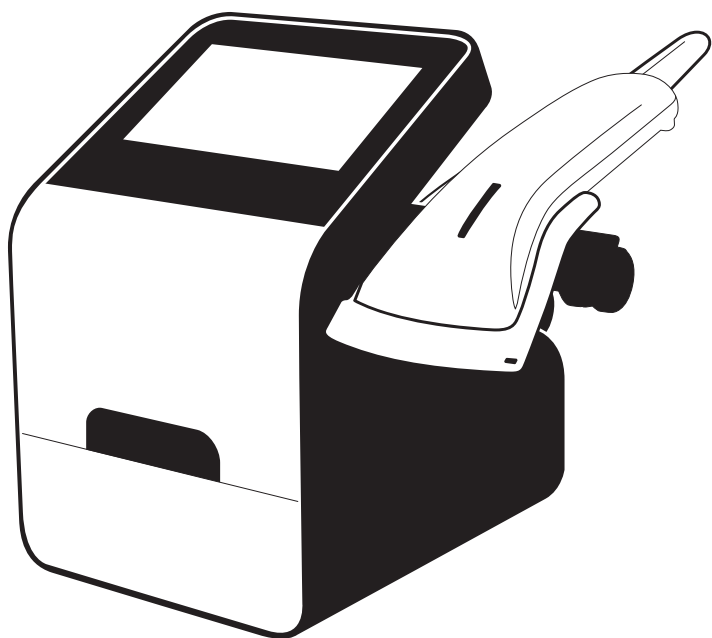
**Abbott**

# **DIGIVAL**™

## SNABBGUIDE

[www.globalpointofcare.eifu.abbott](http://www.globalpointofcare.eifu.abbott)

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---

**REF**    **LFR-000**

**REF**    **LFR-024**

DIGIVAL™

---

**REF**    **L22XWU1-200**

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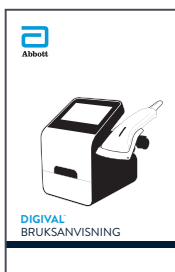
Nätadapter

---

**REF**    **INLFR000**

**REF**    **INLFR024**

Bruksanvisning

---

**REF**    **LFR-003**

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Ange administratörslösenordet

Ange "admin"


The screenshot shows a screen titled 'Inställningar - Adminlösenord' with a blue header. Below the title is the text 'Ange nytt adminlösenord'. The screen displays a virtual keyboard with letters, numbers, and symbols. At the bottom, there are three buttons: 'Tillbaka', a home button, and 'Abc'.

---

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
The screenshot shows a screen titled 'Inställningar - Datum' with a blue header. Below the title is the text 'Välj alternativ'. There are three radio button options: 'DD/MM/ÅÅÅÅ', 'MM/DD/ÅÅÅÅ', and 'ÅÅÅÅ/MM/DD'. To the right of these options are three columns for 'År', 'Månad', and 'Dag', each with a dropdown menu showing '2019', '1', and '24' respectively. At the bottom, there are three buttons: 'Tillbaka', a home button, and 'OK'.

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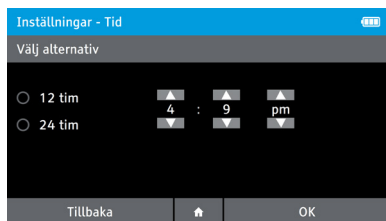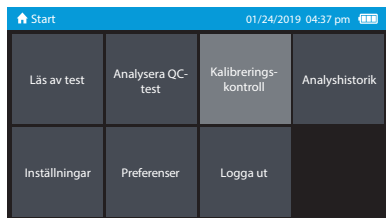LA.TechSupport@abbott.com

**Abbott**

**Abbott Diagnostics Scarborough, Inc.**

10 Southgate Road

Scarborough, Maine 04074 USA

[www.globalpointofcare.abbott](http://www.globalpointofcare.abbott)

[www.globalpointofcare.eifu.abbott](http://www.globalpointofcare.eifu.abbott)

© 2024 Abbott. All rights reserved.

All trademarks referenced are trademarks of either the Abbott group of companies or their respective owners.

INLFR009sv Rev. 7 2024/03

<div>Abbott</div> <div>DIGIVAL</div> <div>QSG, SV</div> <div>Size:</div> <div>5.25 in x 8.5 in</div>	<div>Printed Colors</div> <div><div></div><div>CMYK</div></div>	<div>PN: INLFR009sv</div> <div>Rev: 7</div> <div>Date of Last Revision:</div> <div>7.3 2024/03/15</div>
	<div>Incoming Inspection Colors (For Reference Only)</div> <div>Colors below are not used for printing</div> <div><div></div><div><div>PMS 2995 U Primary Blue</div><div>PMS 303 U Dark Blue</div></div></div>	