

Alere™ NMP22® Test



Caution: U.S.A. Federal law restricts this device to sale and distribution by or on the order of a physician, or to a clinical laboratory, and use is restricted to, by, or on the order of a physician.

INTENDED USE

The Alere NMP22® Test is an enzyme immunoassay (EIA) for the *in vitro* quantitative determination of the nuclear mitotic apparatus protein (NuMA) in stabilized voided urine. The Alere NMP22® Test is indicated as an aid 1) in the diagnosis of persons with symptoms or risk factors for transitional cell cancer (TCC) of the bladder (cut-off ≥ 7.5 U/mL) in conjunction with, and not in lieu of, current standard diagnostic procedures, and 2) in the management of patients with transitional cell carcinoma of the bladder, after surgical treatment to identify those patients with occult or rapidly recurring TCC (cut-off > 10 U/mL).

CONTRAINDICATIONS, WARNINGS and PRECAUTIONS

Contraindications

There are no known contraindications for the Alere NMP22® Test.

Precautions

- For in vitro diagnostic use only.
- Specimens used in this kit must be collected following the Alere NMP22® Urine Collection Kit instructions and stabilized with the NMP22 Stabilizer according to Collection Kit instructions for use.
- Do not mix components from different kit lots.
- Do not use components after the expiration date.
- Use only plastic containers to process or store urine samples, calibrators, sample diluent or laboratory-established validity controls. Do not use any glass containers other than containers in which calibrators and laboratory-established validity controls are supplied.
- Refer to local regulations for the disposal of medical waste when disposing of any remaining kit reagents or specimens.
- Store the Alere NMP22® Urine Collection Kit at a controlled room temperature of 50 to 70°F (10 to 25°C).
- Do not use the Alere NMP22® Urine Collection Kit beyond the expiration date.
- The microtiter plate is sealed in a protective, re-sealable foil pouch. Inspect pouch prior to
 first use and do not use if pouch is damaged or open. If storing unused strips of microtiter
 wells in the re-sealable pouch, inspect pouch prior to each use. Do not use if pouch is
 damaged or was not resealed properly.

Warnings

- Handle all specimens as if capable of transmitting infection.
- · Do not eat, drink, smoke or pipette by mouth where kit reagents are being handled.
- Wear protective laboratory devices such as gloves and laboratory coats when handling specimens and kit reagents; wash hands afterwards.
- The calibrators and sample diluent contain human albumin. Each lot of human albumin is tested and found to be negative for antibody to HIV-1, HIV-2, HbsAg and HCV by FDA approved tests. Because no test method can offer complete assurance that these reagents do not contain HIV, Hepatitis or other infectious agents, reagents should be handled at the BSL 2 as recommended for any potentially infectious human serum or blood specimen. [see CDC-NIH manual; Biosafety in Microbiological and Biomedical Laboratories, 1984 page 12-161.
- The calibrators and sample diluent contain gentamicin sulfate. Danger; may cause an allergic skin reaction, may cause allergy or asthma symptoms or breathing difficulties if inhaled.
- o-Phenylenediamine dihydrochloride (OPD) tablets; Warning: Harmful if swallowed or if
 inhaled, may cause an allergic skin reaction, causes serious eye irritation, suspected of
 causing genetic defects, suspected of causing cancer, very toxic to aquatic life with long
 lasting effects.
- Sulfuric Acid Stop Solution is a known skin and mucous membrane irritant.



- The stabilizer found in the Urine Collection Container of the Alere NMP22® Urine Collection Kit
 and the calibrators and sample diluent found in the Alere NMP22® Test contain trace amounts
 of Fungizone™ and gentamicin as antimicrobial agents.
- The Color Development Buffer, HRP-SAD Reagent and Digoxigenin Anti-NMP22 Reagent contain ProClin® 300. Warning, may cause an allergic skin reaction.
- · Safety Data Sheets for this product are available upon request.
- Follow your national, regional, and local ordinances accordingly for waste disposal regulations.

SUMMARY AND EXPLANATION OF THE TEST

Nuclear matrix proteins (NMPs) make up the internal structural framework of the nucleus^{1,2} and are associated with such functions as DNA replication, RNA synthesis, and hormone binding^{3,4,5}. Further work has indicated that NMPs are involved in regulation and coordination of gene expression^{4,6,7}. Later work by Fey and Penman demonstrated that NMP expression varied with cell type of origin⁸. This observation was followed by work showing that soluble NMPs could be detected in the serum of cancer patients in higher concentrations than were found in normal sera⁸. More recently, Partin and colleagues demonstrated that the pattern of expression of NMP differed in normal prostate tissue, benign prostatic hyperplasia, and prostate cancer. Other work has identified specific NMPs for osteosarcoma, colon and breast cancer.

The antibodies contained in this assay recognize the head domain of NuMA nuclear mitotic apparatus protein. NuMA has been shown to be present in malignant tissues at levels more than ten times higher than in normal tissue. The NuMA antigen moiety detected by the Alere NMP22® Test Kit is referred to as NMP22 by Alere. The assay detects both complexed (>1000 kD) and fragmented (~30 kD) forms of NuMA in stabilized voided urine.

In the urine of healthy individuals, NMP22 antigen is present at low levels. Patients with TCC present in the bladder have been shown to release higher levels of NMP22 antigen into the urine. The assay is designed to quantify NMP22 antigen in stabilized voided urine.

PRINCIPLE OF THE ALERE NMP22® TEST KIT

Alere NMP22® Test is an easy-to-use enzyme immunoassay in a 96-well microplate format. The assay employs two monoclonal antibodies raised against NuMA. Calibrators, laboratory-established internal validity controls and stabilized patient urine samples are reacted with an antibody coated onto wells of the microplate. After washing, the captured NMP22 antigen is reacted with a second antibody labeled with digoxigenin (DIG). After a wash, the digoxigenin-labeled antibody is detected with an anti-digoxigenin antibody coupled to horseradish peroxidase (HRP-SAD) using o-phenylenediamine (OPD) substrate. The reaction is terminated by the addition of 2 molar sulfuric acid (2M H_2 SO4). The concentration of antigen in the urine is directly proportional to the intensity of color developed and the actual concentration is determined from a standard curve. The standard curve is determined by the concurrent testing of the NMP22 Calibrators which nominally range in concentration from 0 to approximately 120 U/mL. The actual calibrator values are printed on the calibrator vial labels.

REAGENTS

Materials Provided in Kit

CAL NMP22 Calibrators

5 x 2 mL (lyophilized)

Contains NMP22 antigen (except cal #1) in a synthetic human urine solution containing human albumin, bovine proteins, and Fungizone and gentamicin as preservatives. Calibrators range in concentration from 0 to ~120 U/mL. The actual concentrations are printed on the vial label.

MT PLATE NMP22 Coated Microplate Strip Well 1 x 96 wells

Each plate consists of twelve 8-well strips coated with mouse monoclonal anti-NMP22 antibody.

DIGOX Digoxigenin Anti-NMP22 Reagent ⟨ 1 x 20 mL
Contains digoxigenin-labeled mouse monoclonal anti-NMP22 antibody with goat, mouse and

Contains horseradish peroxidase labeled sheep anti-digoxigenin with goat, mouse and bovine serum proteins, ProClin300 as a preservative, and other stabilizers.

SAMPLE DIL NMP22 Sample Diluent 1 x 10 mL (lyophilized)
Contains a lyophilized synthetic human urine solution with human albumin, bovine serum proteins, and Fungizone and gentamicin as preservatives.

BUFFER Color Development Buffer (!)

1 x 40 mL Contains hydrogen peroxide in a citrate-phosphate buffer with ProClin300 as a preservative.

WASH CNC 100X Wash Concentrate

1 x 30 mL

Contains a 100X concentrate of a non-ionic detergent in phosphate buffered saline.

OPD Tablets 🚯 🤄 !

4 Tablets

Individual foil-wrapped tablets containing 10 mg of o-phenylenediamine and excipients. Protect from strong light, oxidizing agents, and metals.

INSTRUCTIONS FOR USE

Reagents Required but not Provided

- Alere NMP22® Urine Collection Kit, catalogue number D2100E
- 2 Molar (4 Normal) Sulfuric Acid Stop Solution; see Assay Preparation

Materials not Provided

- 200 µL precision pipette with disposable tips
- Precision pipette with disposable tips to deliver 2 mL
- Microplate reader (490 nm)
- Microplate washer/aspiration device capable of delivering at least 310 µL per well of solution with a 10 second soak between aspirations (optional)
- Centrifuge
- Plastic test tubes to prepare OPD solution
- Plastic test tubes to prepare specimens (centrifuging, or diluting specimens)
- Laboratory-established quality control samples containing a recommended minimum of three levels of NuMa protein

STORAGE INSTRUCTIONS

- Store kit components at 2-8°C.
- Components as packaged are stable through the expiration date printed on the kit box.
- Bring all reagents to room temperature (18-25°C) prior to use.
- Immediately after use, store all reagents at 2-8°C except the diluted wash solution which should be kept at room temperature.
- Reconstituted calibrators and sample diluent are stable for 14 days at 2-8°C or for 30 days when stored at -20°C or lower in single-use aliquots.
- Do not allow aliquots to experience more than one freeze-thaw.
- Keep foil packaging for OPD tablets sealed until ready to use.

SPECIMEN COLLECTION

Patient Preparation

Persons should not be tested with the Alere NMP22® Test if they meet any of the following criteria:

- Have had a total cystectomy.
- Less than five (5) days after an invasive procedure, such as cystoscopy or catheterization.

Specimen Collection

A single void of urine should be collected and STABILIZED IMMEDIATELY by the patient or medical personnel, using the Alere NMP22® Urine Collection Kit, D2100E, Collection should be done between midnight and noon (0:00 to 12:00 hours) if it is for post-surgical patient management, DO NOT USE OTHER METHODS FOR COLLECTING URINE SAMPLES, INVALID NMP22 ANTIGEN MEASURMENTS WILL RESULT FROM IMPROPERLY COLLECTED SAMPLES OR SAMPLES THAT ARE NOT PROPERLY STABILIZED.

- 1. Stabilized urine collected with the Alere NMP22® Urine Collection Kit should be blue/green in color.
- 2. Stabilized samples may remain at room temperature (18-25°C) for up to 96 hours after collection before being processed (see SAMPLE PREPARATION for storage times after processing). After 96 hours they must be stored at 2-8°C. If shipping temperatures are expected to exceed 35°C, samples should be processed and shipped FROZEN on dry ice.
- 3. Sodium azide or other preservatives should not be added to the samples as incorrect results may be obtained when analyzed with Alere NMP22® Test.

ΕN

Sample Preparation

- 1. Process the stabilized urine specimen as follows:
 - a) Ensure that each specimen is at room temperature (18-25°C) prior to processing.
 - b) Centrifuge the total contents of the specimen in plastic tubes at 500 to 1000 x G for 10 to 15 minutes at 10 to 25°C to remove precipitates.
 - c) Decant the supernatant into a separate plastic container.
- 2. Analyze each sample supernatant using the Alere NMP22® Test kit procedure.
- 3. Samples that have been centrifuged and decanted [processed] may be stored at 2-8°C for up to 1 week prior to measurement. Alternatively, samples may be stored at -20°C or lower. -20°C sample storage should be limited to 8 weeks as antigen loss has been reported in some samples after 8 weeks. For storage longer than 8 weeks, -80°C is required. Samples should be thawed at room temperature. Thawing at elevated temperatures (greater than 25°C) may result in loss of antigen activity. Avoid more than three freeze/ thaws.
- Stored samples should be centrifuged and decanted a second time to remove any additional precipitates before testing.

PROCEDURAL NOTES

- · It is important that the user be familiar with the procedure.
- A standard curve must be established with every run.
- Disposable pipette tips should be used to prevent cross-specimen or cross-reagent contamination.
- Avoid interference caused by detergents or other contaminants of labware by thoroughly rinsing prior to use.
- Adherence to protocol incubation times and temperatures is necessary to achieve valid results.
- Avoid microbial contamination of reagents when removing aliquots from the vials. Microbial
 contamination should be suspected if the reagent solutions become cloudy or emit strong
 odor.
- Do not expose OPD reagents to light, any oxidizing agents or metal during storage or incubation.
- Acid Stop Solution is 2 Molar [4 Normal] Sulfuric Acid. Any other concentration of acid
 will produce adverse results. This acid may be purchased commercially or prepared from
 concentrated acid. See ASSAY PREPARATION.
- Do not place samples of urine, controls or calibrators in glass containers other than the containers in which the reagents were supplied.
- Do not allow the microplate to become dry during any part of the assay procedure.

QUALITY CONTROL

Good laboratory practices include the use of control specimens within each assay to ensure that all reagents and procedures are performed properly. It is recommended that each laboratory verify assay performance using a set of controls containing a minimum of three levels of NuMa protein. Because each laboratory may obtain slightly different results, it is suggested that each laboratory sources suitable material and establishes its own range for each level of urine control.

PROCEDURE

Assay Preparation

- Allow kit components and processed stabilized urine samples to equilibrate to room temperature (18-25°C) for 15 to 20 minutes before use.
- · Gently swirl all reagents to ensure mixing prior to use.
- Verify the samples have been prepared as outlined in SAMPLE PREPARATION.
- Prepare Wash Solution from the 100 X Wash Concentrate. Allow 100X Wash Concentrate to reach room temperature (18-25°C). If any precipitate is present, warm the concentrate until the precipitate dissolves.
- Dilute the 100X Wash Concentrate 1:100 with deionized water.

NOTE: 10 mL of concentrate makes 1000mL of wash solution. The vial contains 30 mL.

 Prepare 2M (4 N) Sulfuric Acid Stop Solution by adding 11.1 mL of concentrated sulfuric acid (18 Molar = 36 Normal) to a final volume of 100 mL in deionized water. Store at room temperature.

- Reconstitute each Calibrator with 2 mL deionized water. Reconstitute the Sample Diluent with 10 mL deionized water. Recap and let stand at room temperature (18-25°C) for approximately 10 minutes. Invert and swirl gently, do not vortex. Let stand an additional 10 minutes before use. Be sure the lyophilized material is completely dissolved before use.
- Coated Microplate Strip Well foil package should be opened by cutting close to the package
 edge with the zip-lock strip. Remove strip wells not required from the plate frame. Return
 all unused, coated strip wells to the foil package along with the desiccant packet. Carefully
 reseal the foil package with the zip-lock closure and store at 2-8°C.

Assay Procedure

- 1. Wash NMP22 antibody Coated Strip Well Plate 3 times with the Wash Solution using either an automatic microplate wash aspiration system or manually. For the automatic washing system, settings should allow each well to fill completely (minimum 310 μL) with a 10-second soak prior to aspiration. After the three cycles are completed the wells should be free of residual wash solution. If necessary, tap the plate on absorbent paper. Proceed immediately to step 2.
- Pipet 200 µL/well of each calibrator, laboratory-established validity control and sample into assigned duplicate wells. Incubate 2 hours + 5 minutes at 18-25°C.
- 3. Wash plate 3 times with Wash Solution as described in step 1.
- Pipet 200 µL/well of Digoxigenin Anti-NMP22 Reagent (DIG~22) to all assay wells. Incubate 1 hour ± 2 minutes at 18-25°C.
- 5. Wash plate 3 times with Wash Solution as described in step 1.
- Pipet 200 µL/well of HRP-SAD Reagent to all assay wells. Incubate 30 minutes ± 2 minutes at 18-25°C.
 - NOTE: During the HRP-SAD Reagent incubation, prepare the OPD Solution by transferring the required amount of Color Development Buffer (10 mL for half plate, 20 mL for whole plate) into a plastic test tube. Add one OPD tablet/10 mL of Development Buffer. Cover tube with foil and store in darkened area until ready to use. <u>Prepare this solution no more than 30 minutes before use.</u> The tablet(s) should be completely dissolved before use. Avoid contact of metal objects with this solution.
- 7. Wash plate 3 times with Wash Solution as described in step 1.
- Pipet 200 µL/well of OPD Solution (mix well before use) to each assay well. Incubate 30
 minutes ± 2 minutes at 18-25°C. Place developing plate in a darkened area during color
 development.
- Stop the color development reaction by adding 50µL/well of 2M (4N) H₂SO₄ Stop Solution.
 Tap the plate gently to mix the acid with OPD. Place plate in darkened area at 18-25°C.
- 10. Read the reaction within 10 30 minutes of the addition of the Acid Stop Solution using a plate reading spectrophotometer that has been set at 490nm and blanked to zero absorbance with deionized water. NOTE: To eliminate the possibility of reading errors, the microtiter well used to blank the instrument should only contain deionized water.

RESULTS

Computer Assisted Method

Computer assisted data reduction may be used to calculate results. The performance data in this insert were calculated using a point-to-point curve fit. Curve fitting based on a linear regression analysis has also been validated. It is recommended that each laboratory select either a point-to-point or linear regression curve fit method. Curve fitting based upon other non-linear analysis methods such as log-logit or 4-parameter logistic fit is not recommended. The same curve fit method should be used to calculate results from different assay runs to ensure consistent values are obtained.


Manual Method

The standard curve may be constructed manually on rectilinear graph paper by plotting the mean A490 absorbance value for each set of calibrator replicates on the y-axis versus concentration in U/mL on the x-axis. The best curve fit (straight line) should be drawn through the data points. To determine the NMP22 antigen concentration in a patient sample, find the point on the curve corresponding to the mean absorbance at 490nm of the patient sample and drop a vertical line to the x-axis. Read the concentration of NMP22 antigen in units per milliliter (U/mL).

Elevated Sample Results

If the sample has a mean absorbance at 490nm greater than the mean absorbance of Calibrator #5, the sample should be diluted and re-run. Dilute the sample with the Sample Dilutent so that the value obtained will fall within the range of the calibrator curve. When calculating the actual value of the diluted sample, the NMP22 antigen value obtained in the assay should be multiplied by the dilution factor to obtain the actual NMP22 antigen concentration. For example, if a sample is diluted 1 part sample to 1 part sample dilutent (1 part sample in 2 parts total volume) the dilution factor is 2. If the diluted sample NMP22 antigen value is 80U/mL, the reported value should be 160 U/mL (80 U/mL X 2).

Example Of Standard Curve

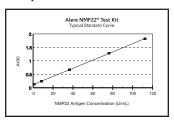

Calibrator	A490				
Unit/mL	Rep 1	Rep 2	Mean	%CV	
0	0.079	0.080	0.080	0.9	
7.5	0.202	0.201	0.202	0.4	
36	0.636	0.629	0.632	0.8	
76	1.253	1.259	1.256	0.3	
112	1.782	1.803	1.792	0.8	

Samples	A490				Units/mL		
	Rep 1	Rep 2	Mean	%CV	Rep 1	Rep 2	Mean
Sample 1	0.122	0.122	0.122	0.0	2.7	2.7	2.7
Sample 2	0.417	0.421	0.419	0.7	21.8	22.1	21.9
Sample 3	1.328	1.382	1.355	2.8	80.8	84.3	82.6

Acceptance Of Results

Calibrators:

The high calibrator (nominal 120 U/mL) should have an absorbance between 0.9 - 2.4.

The lowest calibrator (0 U/mL) should have an absorbance less than 0.15.

Reporting Of Results

NMP22 antigen values within the range of the standard curve may be reported to physicians. If NMP22 antigen values are found to be greater than the high calibrator, it is recommended that the sample be diluted using NMP22 antigen sample diluent and re-assayed. The re-assayed value, corrected for its dilution, should be reported.

Values of ≥7.5 U/mL were found to be optimal for identifying patients with a first occurrence of bladder cancer in the clinical trial reported within this insert. Values of >10 U/mL were found to be optimal for identifying patients at risk for occult or rapidly recurring bladder cancer in the clinical trial reported within this insert.

Do NOT report result when the OD values for the negative control and/or Calibrator 5 are outside of the acceptance limits.

EXPECTED VALUES

Management of TCC of the Bladder (cut-off > 10.0 U/mL)

A prospective clinical trial was performed at 14 institutions and Alere to determine the utility of the Alere NMP22® Test in identifying patients at risk for occult or rapidly recurring transitional cell carcinoma of the bladder (TCC)¹¹. A total of 706 subjects were enrolled; 398 normal healthy volunteers, 117 subjects with benign urologic diseases, 98 subjects with malignancies other than TCC, and 93 subjects with TCC who experienced at least 1 disease episode while enrolled in the trial. A disease episode is defined as the following 1) performance of a surgical procedure for primary or recurring TCC, such as biopsy, fulguration, or trans urethral resection of a tumor, or partial cystectomy or unilateral ureteronephrectomy; 2) collection of a urine sample according to the method described in the Alere NMP22® Urine Collection Kit instructions, between 5 and 60 days after surgical procedure; and 3) performance of a procedure allowing assessment of presence of a neoplasm in bladder, urethra, ureters or pelvis of the kidney, such as cystoscopic examination or total cystectomy, between 2 and 6 months after the surgical procedure. A total of 128 disease episodes occurred among 93 subjects in the trial. To determine sensitivity and

specificity, the disease episodes were classified as negative (no lesions seen on cystoscopy, or if a lesion was seen, pathologic examination of tissue indicated no abnormality present, or atypia or dysplasia), positive (pathologic examination of tissue indicated malignancy present), or unknown (lesion seen but no tissue collected for pathologic examination).

Samples were collected from normal healthy volunteers when each subject had no symptoms of a urologic abnormality and was not under a physician's care for a urologic condition, from subjects with benign urologic conditions when the subject was receiving treatment from a physician for that condition, and from subjects with cancers other than TCC when the subject was being treated or followed for that mailignancy by a physician.

The percent distribution of NMP22 antigen levels in self-referred healthy subjects, patients under treatment for benign diseases, and patients with malignancies of other sites is presented in the following table. The malignancy group includes those with active disease under treatment and those with malignancy diagnosed within the previous 18 months but which was no longer clinically evident.

Percent Distribution of NMP22 Antigen (U/mL)							
	N	0-10	>10-20	>20-50	>50-100	>100	
Healthy Subjects							
Male ≥ 50	215	94.9%	3.3%	1.9%	0%	0%	
Female ≥ 50	151	87.3%	6.6%	4.6%	0.7%	0.7%	
< 50 years both sexes	32	90.6%	9.4%	0%	0%	0%	
Total	398	91.7%	5.0%	2.8%	0.3%	0.3%	
Benign Disease*		,					
UTI/Cystitis	26	84.6%	11.5%	3.8%	0%	0%	
Urinary Calculi	16	93.8%	0%	0%	6.3%	0%	
BPH/Prostatitis	52	92.3%	7.7%	0%	0%	0%	
Other	37	83.8%	5.4%	8.1%	2.7%	0%	
Total	117	88.0%	7.7%	3.4%	0.9%	0%	
Cancers other than TCC							
Head and Neck	6	83.3%	0%	16.7%	0%	0%	
GI Tract	12	83.3%	0%	8.3%	0%	8.3%	
Cardiovascular Pulmonary	12	58.3%	8.3%	16.7%	16.7%	0%	
Leukemia/ Lymphoma	11	63.6%	18.2%	9.1%	9.1%	0%	
Prostate	22	81.8%	0%	13.6%	4.5%	0%	
Kidney (non-TCC)	18	77.8%	11.1%	11.1%	0%	0%	
Other**	17	64.7%	17.6%	5.9%	0%	11.8%	
Total	98	73.5%	10.2%	9.2%	4.1%	3.1%	

^{*} Some patients are included in more than one category.

In this study 92% of the healthy subjects had NMP22 antigen concentrations of 10.0 U/mL or lower. Each laboratory should establish its own reference value.

Of the total 128 disease episodes (See EXPECTED VALUE for description of a disease episode) 116 could be classified as negative (no evidence of malignancy) or positive (occult or rapidly recurring malignant disease present). The first disease episode for each TCC subject in the trial was also analyzed; among 93 subjects, 87 had first disease episodes that could be classified as negative or positive. The following tables show the Alere NMP22® Test results relative to a reference value of >10.0 U/mL for patients with occult or rapidly recurring TCC, following surgical treatment for TCC, and for patients with no malignant disease present.

These tables and analyses are broken down into the following categories: (1) all episodes of disease (multiple events per patient), (2) first episode of disease (3) sexes combined, (4) sexes separately, due to differences in NMP22 Test values seen between the sexes.

^{**}The 'other' category in the cancers other than TCC included malignancies of the bones, joints, cartilage, breast, uterine cervix, other female organs, testes and thyroid, pheochromocytoma, and hemangiopericytoma of the lea.

Sexes Combined; All Disease Episodes (cystoscopic exam 2 to 6 months after Surgical Treatment)						
NMP22 (U/mL) Value	Occult/Rapidly Recurring Malignant Disease Present	No Evidence of Malignant Disease	Total			
≤ 10 U/mL	10	64	74			
> 10 U/mL	24	18	42			
Total	34	82	116			
		95% confider	nce level			
Sensitivity	70.6%	55.3 - 8	5.9			
Specificity	78.0%	69.0 - 87.0				
Accuracy	75.9%	68.1 - 83.7				
Positive Predictive Value	57.1%	42.1 - 72.1				
Negative Predictive Value	86.5%	78.7 -94.3				

Sexes Combined; First Disease Episodes (cystoscopic exam 2 to 6 months after Surgical Treatment)						
NMP22 (U/mL) Value	Occult/Rapidly Recurring Malignant Disease Present	No Evidence of Malignant Disease				
≤ 10 U/mL	6	46	52			
> 10 U/mL	19	16	35			
Total	25	62	87			
		95% confic	lence level			
Sensitivity	76.0%	59.3 -	- 92.7			
Specificity	74.2%	63.3 -	- 85.1			
Accuracy	74.7%	65.6 - 83.8				
Positive Predictive Value	54.3%	37.8 - 70.8				
Negative Predictive Value	ative Predictive Value 88.5% 79.8 - 97.2					

Males; All Disease Episodes (cystoscopic exam 2 to 6 months after Surgical Treatment)						
NMP22 (U/mL) Value	Occult/Rapidly Recurring Malignant Disease Present	No Evidence of Malignant Total Disease				
≤ 10 U/mL	8	58	66			
> 10 U/mL	19	11	30			
Total	27	69	96			
		95% confidence level				
Sensitivity	70.4%	53.2	- 87.6			
Specificity	84.1%	75.5 - 92.7				
Accuracy	80.2%	72.2 - 88.2				
Positive Predictive Value	63.3%	46.1 - 80.5				
Negative Predictive Value	87.9%	80.0 - 95.8				

Males; First Disease Episodes (cystoscopic exam 2 to 6 months after Surgical Treatment)							
NMP22 (U/mL) Value	Occult/Rapidly Recurring Malignant Disease Present	No Evidence of Malignant Disease	Total				
≤ 10 U/mL	4	43	47				
> 10 U/mL	16	11	27				
Total	20	54	74				
		95% confide	ence level				
Sensitivity	80.0%	62.5 -	97.5				
Specificity	79.6%	68.9 - 90.3					
Accuracy	79.7%	70.5 - 88.9					
Positive Predictive Value	59.3%	40.8 - 77.8					
Negative Predictive Value	91.5%	83.5 -	00.5				

Females; All Disease Episodes (cystoscopic exam 2 to 6 months after Surgical Treatment)						
NMP22 (U/mL) Value	Occult/Rapidly Recurring Malignant Disease Present	No Evidence of Malignant Disease	Total			
≤ 10 U/mL	2	6	8			
> 10 U/mL	5	7	12			
Total	7	13	20			
		95% confidence level				
Sensitivity	71.4%	37.9	- 100			
Specificity	46.2%	19.1 - 73.3				
Accuracy	55.0%	33.2 - 76.8				
Positive Predictive Value	41.7%	13.8 - 69.6				
Negative Predictive Value	75.0%	45.0	- 100			

Females; First Disease Episodes (cystoscopic exam 2 to 6 months after Surgical Treatment)						
NMP22 (U/mL) Value	Occult/Rapidly Recurring Malignant Disease Present	No Evidence of Malignant Disease				
≤ 10 U/mL	2	3	5			
> 10 U/mL	3	5	8			
Total	5	8	13			
		95% confid	lence level			
Sensitivity	60.0%	17.1 -	- 100			
Specificity	37.5%	4.0 -	71.0			
Accuracy	46.2%	19.1 - 73.3				
Positive Predictive Value	37.5%	4.0 - 71.0				
Negative Predictive Value	60.0%	17.1 -	- 100			

As indicated by the above statistical analysis, urinary NMP22 test values of greater than 10.0 U/mL for samples collected following a surgical procedure may indicate occult or rapidly recurring malignant disease of the bladder. Patients with NMP22 test values equal to or below 10.0 U/mL are less likely to have malignant disease on follow-up two to six months later.

Sensitivity, Specificity, PPV and NPV for Management of Patients by Range of Incidence Rate						
Incidence Rate	Sensitivity	Specificity	PPV	NPV		
10%	76.0%	74.2%	24.7%	96.5%		
20%	76.0%	74.2%	42.4%	92.5%		
30%	76.0%	74.2%	55.8%	87.8%		

Diagnosis of TCC of the Bladder (cut-off ≥ 7.5 U/mL)

A second prospective clinical trial was performed at 33 sites to determine the utility of the Alere NMP22® Test as an aid in diagnosing transitional cell carcinoma of the bladder (TCC). Voided urine samples were collected from a total of 1147 individuals: 769 patients with unresolved hematuria or other symptoms or risk factors for bladder cancer (e.g. dysuria, exposure to carcinogens, history of smoking), 329 self-referred, hematuria negative, normal healthy volunteers, and 49 patients with active cancers other than those of the urinary tract.

To determine sensitivity and specificity, patients with risk factors for bladder cancer were classified as positive or negative for TCC. Patients were considered negative for TCC if their evaluation included a negative voided cytology, cystoscopy and upper tract evaluation (such as IVP or ultrasound). A negative voided cytology was defined as one in which no malignant or dysplastic cells were identified. A result of suspicious cells required further evaluation until the diagnosing physician deemed that no further diagnostic procedures were necessary at that time. Negative cystoscopy and upper tract evaluations were defined as those in which no tumor was identified, or if identified, was pathologically confirmed as non-malignant. Patients were considered positive for TCC if they had a positive cytology and/or cystoscopy and/or upper tract diagnostic procedure. A positive cytology was defined as one in which malignant or dysplastic cells were present. A result of suspicious cells required further evaluation. A positive cystoscopy was defined as one in which a tumor was seen endoscopically, and for which there was pathologic confirmation of TCC of biopsied or resected tissue. A positive upper tract evaluation was defined as one in which a tumor, filling defect, or wall thickening was identified and there was pathologic confirmation of malignancy of biopsied or resected tissue. No patients were found positive for upper tract cancer.

A single voided urine sample was collected from each patient with symptoms or risk factors for bladder cancer during their standard diagnostic evaluation. Samples from normal healthy volunteers were collected when each subject had no symptoms of a urologic abnormality, and had no history of a urologic disease during the prior twelve months. Samples were tested for hematuria by dipstick to rule out undiagnosed disease. Only samples that were negative for blood were included in the analysis. Samples from the patients with other cancers were collected when patients had clinically or pathologically confirmed malignancy and were not undergoing chemo-, immuno-, or radiation therapy at the time of collection. These patients must not have been diagnosed with a urinary tract disease within the prior twelve months.

The percent distribution of NMP22 antigen levels in hematuria-negative healthy subjects, persons with risk factors for TCC newly diagnosed with benign disease (as yet untreated), and persons diagnosed with cancers other than the bladder and not yet receiving treatment, is presented in the following table.

Percent Distribution of NMP22 Levels (U/mL)								
	N	0 -<7.5	7.5-10	>10-20	>20-50	>50-100	>100	
Healthy Subjects								
Male > 50	111	88.3%	5.4%	4.5%	0.9%	0.9%	0%	
Female > 50	218	85.8%	5.5%	6.0%	2.8%	0%	0%	
Total	329	86.6%	5.5%	5.5%	2.1%	0.3%	0%	
Benign Disease								
UTI/Cystitis	58	69.0%	12.1%	8.6%	5.2%	5.2%	0%	
Urinary Calculi	71	64.8%	8.5%	8.5%	9.9%	4.2%	4.2%	

Percent Distribution of NMP22 Levels (U/mL)								
	N	0 -<7.5	7.5-10	>10-20	>20-50	>50-100	>100	
BPH/Prostatitis	164	81.7%	4.9%	7.3%	2.4%	1.8%	1.8%	
Other Benign Conditions	259	78.0%	7.7%	7.3%	2.3%	1.5%	3.1%	
Total Benign Disease*	448	77.2%	7.8%	7.4%	3.3%	1.3%	2.9%	
Other Cancers								
GI Tract	11	100%	0%	0%	0%	0%	0%	
Leukemia/ Lymphoma	5	80.0%	20%	0%	0%	0%	0%	
Prostate	21	90.5%	4.8%	0%	4.8%	0%	0%	
Renal	1	0%	0%	0%	100%	0%	0%	
Ovarian/Cervical	11	90.9%	9.1%	0%	0%	0%	0%	
Total Other Cancers	49	89.8%	6.1%	0%	4.1%	0%	0%	

^{*} Some patients are included in more than one category.

The following table shows the percent distribution of the Alere NMP22® Test results for final diagnoses of the 769 patients who had symptoms or risk factors for TCC.

Percent	Percent Distribution of NMP22 Levels (U/mL)							
		N	0-<7.5	7.5-10	>10 -20	>20 -50	>50- 100	>100
Risk Factor	No Urinary Tract Disease	265	79.2%	8.3%	9.1%	2.6%	0.4%	0.4%
Patients	Benign Urinary Tract Disease	448	77.2%	7.8%	7.4%	3.3%	1.3%	2.9%
	TCC	56	37.5%	10.7%	12.5%	16.1%	7.1%	16.1%

The following table shows sensitivity and specificity of the Alere NMP22® Test for this study for TCC using a cut-off of ≥7.5 U/mL.

NMP22 Res	NMP22 Results: Sensitivity, Specificity, PPV and NPV for Risk Factor Patients (T0-T4)				
	Sensitivity	Specificity	PPV	NPV	
	(95% Exact CI)	(95% Exact CI)	(95% Exact CI)	(95% Exact CI)	
NMP22*	62.5%	78.0%	18.2%	96.4%	
(cut-off ≥	(35/56)	(556/713)	(35/192)	(556/577)	
7.5 U/mL)	(48.5-75.1%)	(74.8-81.0%)	(13.0-24.4%)	(94.5-97.7%)	

^{*}Compared to result of all three tests (cystoscopy, voided cytology, imaging):
Positive= positive on at least one of the three tests. Negative=negative on all three tests.

The following table shows the sensitivity and specificity of voided urine cytology for this study for TCC.

Voided Cytology: Sensitivity, Specificity, PPV and NPV for Risk Factor Patients (T0-T4)				
	Sensitivity (95% Exact CI)	Specificity (95% Exact CI)	PPV (95% Exact CI)	NPV (95% Exact CI)
Voided Cytology*	32.6% (15/46) (19.5-48.0%)	100% (713/713) (99.5-100%)	100% (15/15) (78.2-100%)	95.8% (713/744) (94.1-97.2%)

^{*}Compared to result of all three tests (cystoscopy, voided cytology, imaging):

Positive= positive on at least one of the three tests; Negative = negative on all three tests. Not

every patient positive for TCC had a cytology result, but every patient negative for TCC did have
a cytology result.

The following tables show the NMP22 test results (cut-off ≥7.5 U/mL) compared to cytology results and the combination of NMP22 results and cytology for the different stages and grades of TCC.

Percent and (95% confid	Percent and Fraction of Positives by stage (T0 ¹² -T4) (95% confidence interval)			
	NMP22 Level Cut-off greater than or equal to 7.5 U/mL	Voided Cytology	NMP22 & Cytology Results Combined*	
T0 ¹²	60.0%	0%	60.0%	
	(3/5)	(0/5)	(3/5)	
	(14.7-94.7%)	(-)	(14.7-94.7%)	
Та	45.0%	16.7%	57.9%	
	(9/20)	(3/18)	(11/19)	
	(23.1-68.4%)	(3.6-41.4%)	(33.5-79.8%)	
Tis	80.0%	66.7%	100%	
	(4/5)	(2/3)	(4/4)	
	(28.4-99.5%)	(9.4-99.2%)	(39.8-100%)	
T1	63.6%	50.0%	72.7%	
	(7/11)	(5/10)	(8/11)	
	(30.8-89.1%)	(18.7-81.3%)	(39.0-94.0%)	
T2, T3, T4	76.9%	55.6%	92.3%	
	(10/13)	(5/9)	(12/13)	
	(46.2-95.0%)	(21.1-86.3%)	(64.0-99.8%)	
Tx	100%	0%	100%	
	(2/2)	(0/1)	(2/2)	
	(15.8-100%)	(-)	(15.8-100%)	

^{*} Positive= positive on either test: Negative= negative on both tests

Percent and Fraction of Positives by Grade (95% confidence interval)				
	NMP22®Test	Voided	NMP22 & Cytology	
	Cut-off ≥7.5 U/mL	Cytology	Results Combined*	
No malignancy	60.0% (3/5) (14.7-94.7%)	0% (0/5) (-)	60.0% (3/5) (14.7-94.7%)	
Low	50.0%	13.3%	56.3%	
	(9/18)	(2/15)	(9/16)	
	(26.0-74.0%)	(1.7-40.5%)	(29.9-80.3%)	
Medium	70.6%	42.9%	82.4%	
	(12/17)	(6/14)	(14/17)	
	(44.0-89.7%)	(17.7-71.1%)	(56.6-96.2%)	
High	68.8%	58.3%	87.5%	
	(11/16)	(7/12)	(14/16)	
	(41.3-89.0%)	(27.7-84.8%)	(61.7-98.5%)	

^{*} Positive= positive on either test; Negative= negative on both tests

Conclusion

Urinary NMP22 test values equal to or greater than 7.5 U/mL in patients with symptoms or risk factors for bladder cancer may indicate the presence of TCC of the bladder. Patients with NMP22 test values below 7.5 U/mL are less likely to have TCC.

Sensitivity, Specificity, PPV and NPV for Diagnosis of Patients by Range of Incidence Rates				
Incidence Rate	Sensitivity	Specificity	PPV	NPV
1%	62.5%	78.0%	2.8%	99.5%
7.0%	62.5%	78.0%	17.6%	96.5%
7.3% (actual rate)	62.5%	78.0%	18.2%	96.4%
15.0 %	62.5%	78.0%	33.4%	92.2%

Urine NMP22 antigen concentrations should not be interpreted as evidence of the presence or absence of malignant disease in the bladder without corroboration from other diagnostic procedures. Other clinically accepted tests and procedures should be considered in the diagnosis of disease and good patient management.

PERFORMANCE CHARACTERISTICS

Limit of Detection

The lowest concentration of NMP22 antigen that can be measured reliably with the Alere NMP22® Test Kit is 2.1 U/mL. The minimal detectable level is defined as that NMP22 antigen value which corresponds to the absorbance that is two standard deviations (2SD) above the mean absorbance of twenty replicate determinations of calibrator #1 (0 U/mL).

Precision

Following procedures outlined in the National Committee for Clinical Laboratory Standards (NCCLS) Document EP5-A; Evaluation of Precision Performance of Clinical Chemistry Devices, within-run and total precision were evaluated for three internally established urine controls and five patient specimens. The specimens were assayed in duplicate in each of two independent runs repeated daily over a 20-day period.

Specimen	Number	Mean (U/mL)	Within Run % CV	Total % CV
Urine control 1	80	7.0	4.3	7.9
Urine control 2	80	25.8	2.5	4.9
Urine control 3	80	51.4	2.2	3.7
Specimen A	80	6.3	5.0	12.4
Specimen B	80	16.5	3.2	6.7
Specimen C	80	31.1	2.5	5.8
Specimen D	80	63.0	2.3	5.4
Specimen E	80	96.3	2.3	5.7

Recovery

Known concentrations of NMP22 antigen from stabilized patient urine were added to stabilized urine containing low endogenous levels of NMP22 antigen. The samples were measured in duplicate. The mean of 2 assays is reported. Mean recoveries of NMP22 antigen in stabilized urine ranged from 89% to 111% with an overall mean of 99%. An example of a typical recovery study is summarized below:

NMP22 Antigen Added (U/mL)	NMP22 Antigen Recovered (minus endogenous)	Percent Recovery
48.6	45.9	94
31.5	28.4	90
23.5	22.3	95
11.0	11.4	103
	Mean Recovery	96

Linearity Of Dilution

Six (6) stabilized urine samples containing elevated NMP22 antigen levels were serially diluted with the NMP22 Sample Diluent (NMP22 Urine Calibrator 1 (0 U/mL)) and assayed in triplicate. Linear regression analysis of the NMP22 antigen concentrations versus dilution was performed. The slopes for the 6 samples ranged from 0.80 to 1.06 with a correlation coefficient of greater than 0.993, thus demonstrating that the samples will dilute linearly.

Potentially Interfering Substances

Substances listed were evaluated and found to have no significant effects on the results of the Alere NMP22® Test Kit at the following concentrations:

Substance	Concentration
Urine Analytes	
Protein (Human serum albumin)	100 mg/dL
Protein (Human IgG)	100 mg/dL
Hemoglobin	1.6 mg/dL
Red Blood Cells (#/dL)	1.8 x 10 ¹¹
Whole Blood (v/v%)	1.0%
Glucose	20.0 mg/dL
Cytokeratins (TPA)(U/dL)	40 U/dL
Therapeutic Agents	
BCG (Tice)(CFU/mL)	5.0 x 10 ⁶
Thiotepa	60.0 mg/dL
Drugs	
Digoxin	0.05 mg/dL
Acetaminophen	20 mg/dL
Sodium Ascorbate	20 mg/dL
Caffeine	20 mg/dL
Sodium Salicylate	20 mg/dL
Sodium Acetylsalicylate	20 mg/dL
Ampicillin	20 mg/dL
Tetracycline	20 mg/dL

LIMITATIONS

- NMP22 antigen concentrations should not be interpreted as evidence of the presence or absence of malignant disease in the bladder without corroboration from other diagnostic procedures and should only be used in conjunction with other diagnostic information in the management or diagnosis of patients with transitional cell carcinoma of the bladder.
- Patients with known malignancy of the bladder or transitional cell carcinoma in other parts of the urinary tract may have urinary NMP22 antigen levels within the range of concentrations observed in individuals with no known malignancy.
- For increased ability to accurately detect the NMP22 antigen, clinical studies investigating the use of the Alere NMP22° Test in the management of post-surgical TCC patients have shown that the urine sample must be collected between the hours of midnight and noon (0:00 to 12:00 hours)¹º. The clinical trial investigating the use of the Alere NMP22° Test in the initial diagnosis of patients with TCC did not restrict the collection time. For details on urine collection refer to Alere NMP22° Urine Collection Kit Instructions for Use.
- Elevated urinary NMP22 antigen levels have been observed in individuals with no known malignancy of the urinary tract. Some published studies have also shown that patient samples with gross hematuria may cause elevated urinary NMP22 antigen levels. Occasional elevations have been observed immediately after extreme exercise (e.g. running more than 10 miles) in apparently healthy individuals, in some benign conditions (e.g. interstitial cystitis, urinary tract infections), in patients with renal cancer and malignancy of any site undergoing systemic chemotherapy. Elevated values are always seen in patients who have undergone total cystectomy. Significance of these elevated results is unknown. Physicians should use some judgment in determining when samples are collected.
- Samples collected fewer than 5 days after an invasive procedure such as cystoscopy or catherization of the urethra may result in elevated values due to tissue damage.

- Samples collected while the patient is undergoing intravesical therapy may not accurately
 reflect the presence or absence of malignancy in the bladder. Interpretation of NMP22 test
 results from these samples has not been adequately determined.
- Only urine that has been stabilized with the Alere NMP22® Urine Stabilizer should be used in this assay. Unstabilized urine and other body fluids should not be used in this test.

REFERENCES

- Berezney R and DS Coffey. 1974. Identification of a nuclear protein matrix. Biochem BiophysRes Commun, 60:1410.
- Fey EG, G Krochmalnic, and S Penman. 1986. The non-chromatin substructures of the nucleus: the ribonucleoprotein (RNP)-containing and RNP-depleting matrices analyzed by sequential fractionation and resinless section electron microscopy. J Cell Biol, 102:1654.
- Pardoll DM, B Vogelstein, and DS Coffey. 1980. A fixed site of DNA replication in eukaryotic cells, Cell, 19:527.
- Zeitlin S, A Parent, S Silverstein, and A Efstratiadis. 1987. Pre-mRNA splicing and the nuclear matrix. Mol Cell Biol. 7:111.
- Kumara-Siri MH, LE Shapiro, and MI Surks. 1986. Association of the 3,5,3e-triiodo-L-thyronine nuclear receptor with the nuclear matrix of cultured growth hormone-producing rat pituitary tumor cells (GC cells). J Biol Chem, 261:2844.
- Nakayasu H and R Berezney. 1989. Mapping replicational sites in the eukryotic nucleus. J Cell Biol, 108:1.
- Berrios M, N Osheroff and PA Fisher. 1985. In situ localization of topoisomerase II, a major polypeptide component of the Drosophila nuclear matrix fraction. Proc Natl Acad Sci USA, 82:4142.
- Fey EG, and S Penman. 1988 Nuclear matrix proteins reflect cell type of origin in cultured human cells. Proc Natl Acad Sci USA, 85:121.
- Miller TE, LA Beausang, LF Winchell and GP Lidgard. 1992. Detection of nuclear matrix proteins in serum from cancer patients. Cancer Res, 52:422.
- Chen YT, CL Hayden, KJ Marchand and RW Makuch. 1997. Comparison of Urine Collection Methods for Evaluating Urinary Nuclear Matrix Protein, NMP22, as a Tumor Marker. J. Urol, 158:1890
- Soloway MS, JV Briggman, GA Carpinito, GW Chodak et al. 1996. Use of a new tumor marker, urinary NMP22, in the detection of occult or rapidly recurring transitional cell carcinoma of the urinary tract following surgical treatment. J Urol, 156:363.
- Epstein JI, et al. The World Health Organization/International Society of Urological Pathology Consensus Classification of Urothelial (Transitional Cell) Neoplasms of the Urinary Bladder. Am J. of Surg. Pathol. 22(12):1435-1448; 1998.

ORDERING and CONTACT INFORMATION D1100E Alere NMP22® Test



US: 1-877-441-7440 OUS: 1-321-441-7200

TECHNICAL SUPPORT

Advice Line

Further information can be obtained from your distributor, or by contacting Alere $^{\text{TM}}$ Technical Support on:

US

+1 877 866 9356 TS.SCR@alere.com

Africa, Russia, CIS

+972 8 9429 683 ARCISproductsupport@alere.com

Asia Pacific

+61 7 3363 7711 APproductsupport@alere.com

Canada

+1 800 818 8335 Canproductsupport@alere.com

Europe & Middle East

+44 161 483 9032 EMEproductsupport@alere.com

Latin America

+57 2 6618797 LAproductsupport@alere.com







Hazard Pictogram. See Warnings.



Alere Scarborough, Inc. 10 Southgate Road Scarborough, Maine 04074 USA www.alere.com

© 2022 Alere. All rights reserved.

The Alere Logo, Alere and NMP22 are trademarks of the Alere group of companies. All trademarks referenced are trademarks of their respective owners.

L0220 Rev. 9 2022/03