

SD BIOLINE HIV Ag/Ab Combo

NOTE:
Each test can be used **ONLY ONE TIME**.
Do not try to use the test more than once.



Check the expiry date.
If expiry date has passed, use another kit.



Put on the gloves.
Use new gloves for each patient



Open the test pouch and **write the patient's name** on the test device.


W/B

S/P

Preparation



Overview of the Device

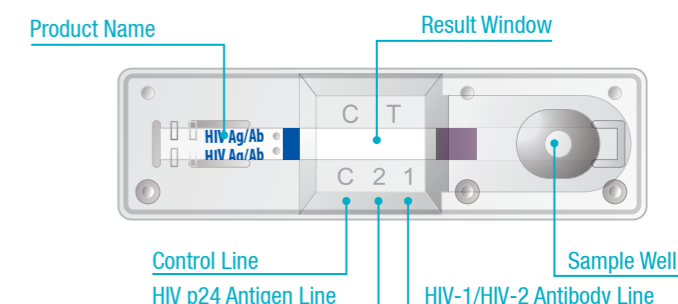

Whole Blood Specimen



Prick the patient's finger with lancet.
Discard the lancet pierced to the cap in the biohazard bag after pricking.




Use the disposable dropper to **collect the blood**. (up to fill line)



Add 1 drop of assay diluent into the sample well.



Wait 20 minutes.
Don't read test results after 20 minutes.
Reading too late can yield false results.

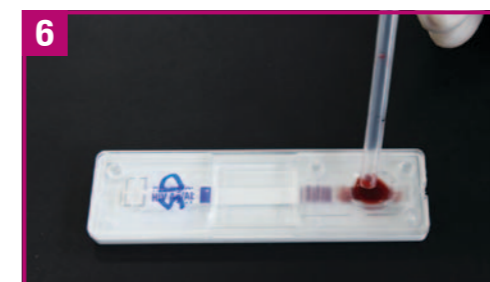

Add drawn of blood into the round sample well ('S' marked).
Discard the pipette in the biohazard bag after adding blood.



Wait 1 minute after adding sample.

IMPORTANT CAUTION



Plasma or Serum Specimen



Prepare the serum/plasma samples.



Use the disposable dropper to collect serum/plasma (Up to fill-line).



Add serum/plasma drawn by disposable dropper into the sample well ('S' marked).




Wait 20 minutes.
Don't read test results after 20 minutes.
Reading too late can yield false results.

Interpretation

NEGATIVE

Only the control line "C" appear in the result window.

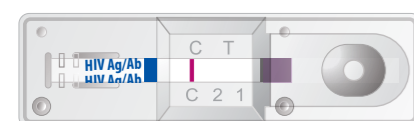


POSITIVE

[If any test band is appeared, interpret it as positive]


HIV-1/HIV-2 Antibody Positive
When 2 lines appear "C", "1" line



HIV p24 Antigen positive
When 2 lines appear "C", "2" line

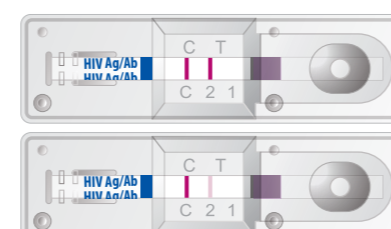

HIV p24 Antigen and HIV-1/HIV-2 Antibody Positive
When 3 lines appear "C", "1", "2" line

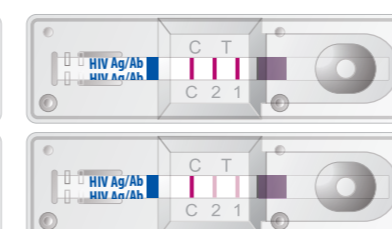

INVALID

No control line "C" appears in the result window.
It is recommended that the specimen be re-tested.



Tel : +82-31-899-2800
Fax : +82-31-899-2840
E-mail : sales@standardia.com

© 2015 Alere. All rights reserved. The Alere Logo and Alere are trademarks of the Alere group of companies.
QU-03FK30EN / Rev. 1506-01

www.standardia.com