

Quick Reference Guide


SD BIOLINE Dengue Duo

Dengue NS1 Ag + Ab Combo

Preparation



Test pouch
(Test Device)

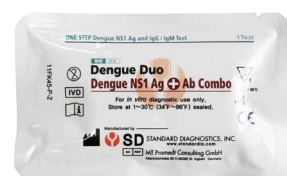

Assay diluent
(for Dengue IgG/IgM)



Dengue NS1 Ag

Disposable Dropper (100µl)



Dengue IgG/IgM

Disposable capillary pipette (10µl)



Timer
(Not Provided)



Disposable Gloves
(Not Provided)




Micropipette & Disposable Tip
(Not Provided)



Check Point !

Capillary Pipette

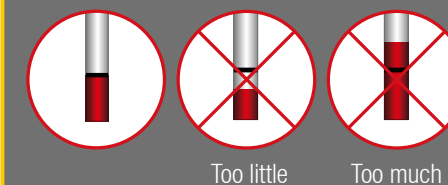







Assay diluent



Hold the assay diluent vertically.

Overview of the Device



Dengue NS1 Ag Dengue IgG/IgM

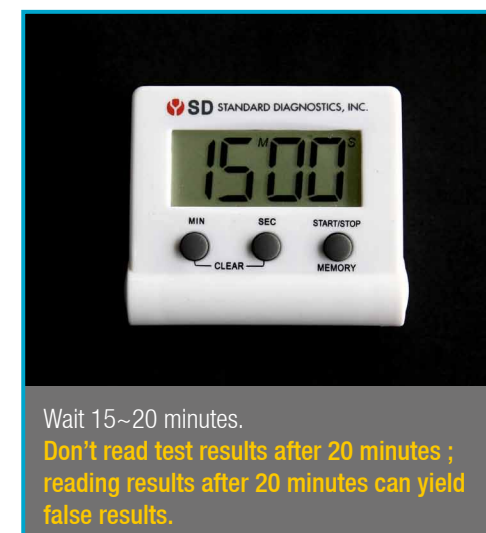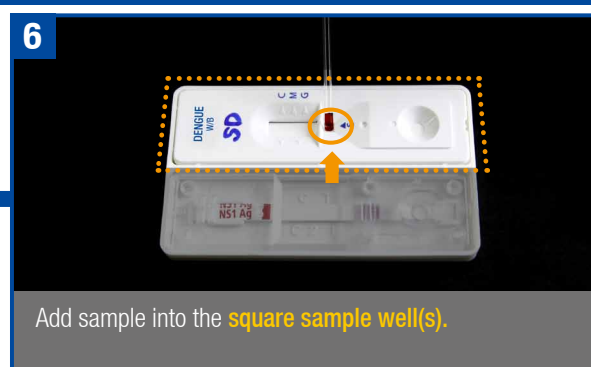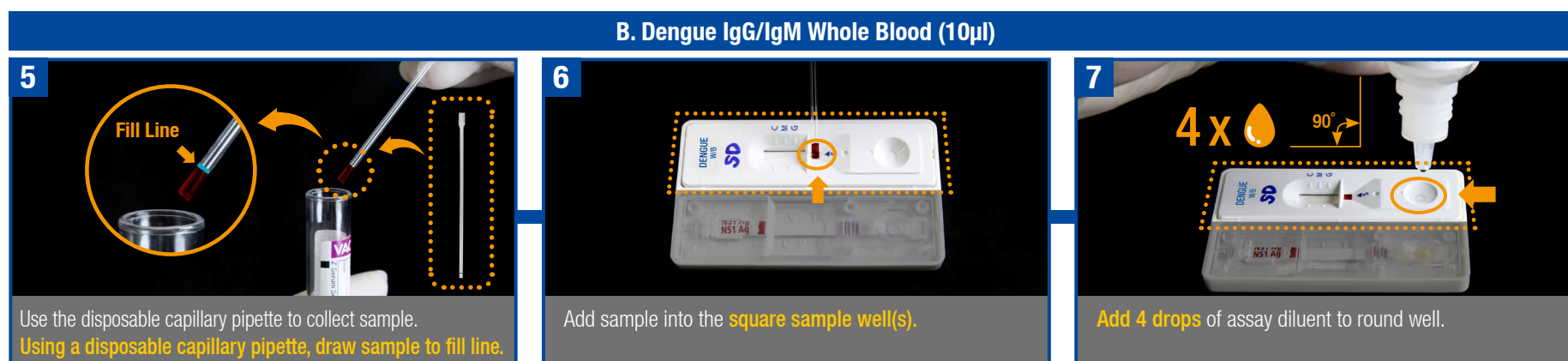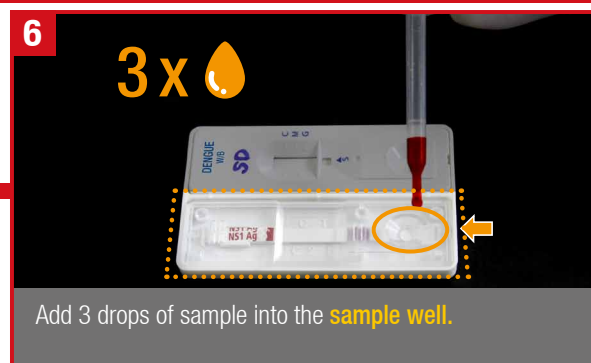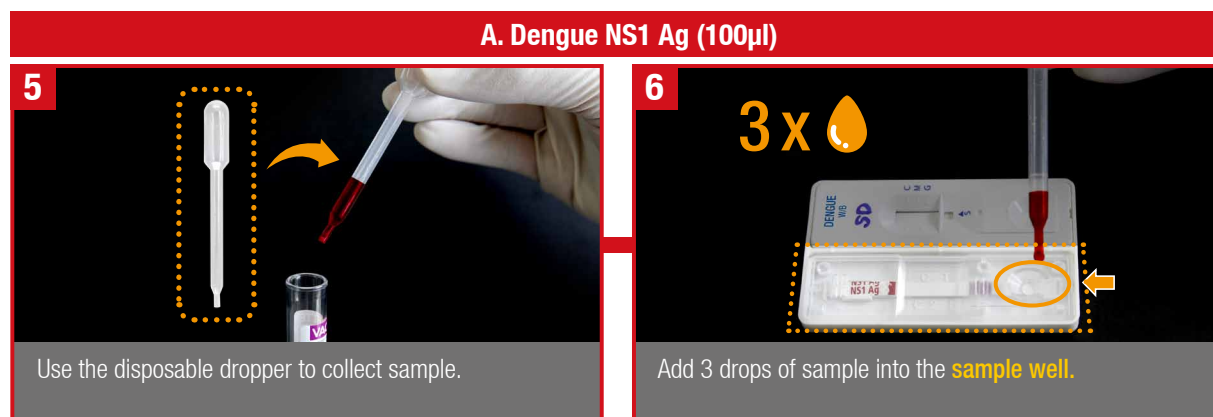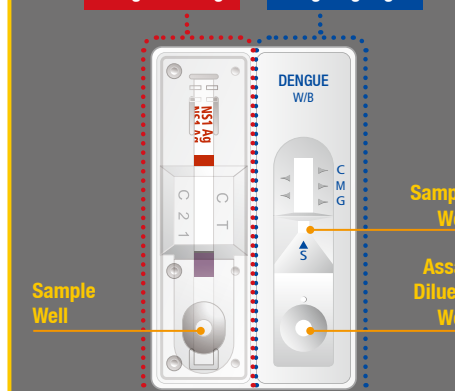

Caution:
Each test can be used **ONLY ONE TIME**.
Do not reuse the test device.


Interpretation

POSITIVE ❖ If any of NS1, IgM, IgG test band show positive results, please consider as a positive result.

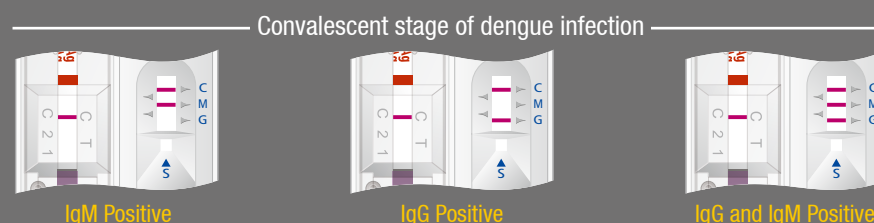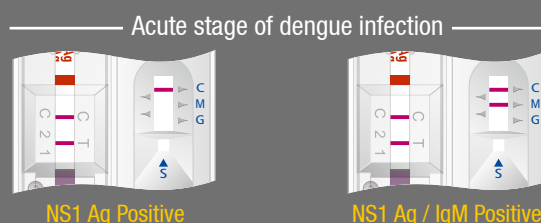

IgG and IgM Positive

NEGATIVE

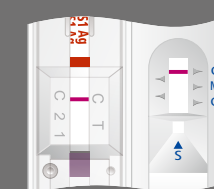


INVALID RESULT

Dengue NS1 Ag



Dengue IgG/IgM

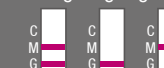

- No control line "C" in result window
- It is recommended that the specimen be retested using a new test device.



Tel : +82-31-899-2800
Fax : +82-31-899-2840
E-mail : sales@standardia.com

© 2014 Alere. All rights reserved. The Alere Logo and Alere are trademarks of the Alere group of companies.

QU-11FK45EN / Rev. 1411-01

www.standardia.com