

DETERMINE™ HBsAg 2

QUICK REFERENCE GUIDE


Before you begin

- Please read the Instructions for Use in its entirety, prior to use.
- Gather materials you will need.
- Cover your work space with a clean, disposable absorbent workspace cover.
- Put on your disposable gloves.
- Chase Buffer must be brought to room temperature (between 15-30°C) prior to testing.

For more detailed instructions, please refer to the Instructions for Use.


1 Prepare Test

Tear one strip from the right and remove cover.

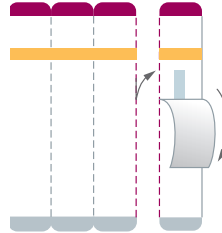

2 Place Test

Place one strip on a flat surface where the test is to be performed.

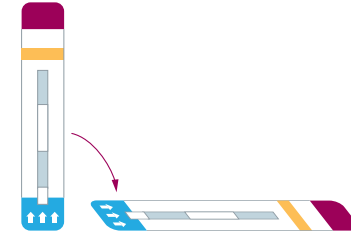

3 Adding Sample Type 1

Venipuncture Whole Blood
Add 50 µL of whole blood (precision pipette) to the middle of the Sample Pad. Wait 1 minute and add one drop of Chase Buffer.



Adding Sample Type 2

Fingerstick Whole Blood
Add 50 µL of whole blood to the Sample Pad. When all the blood is transferred from the capillary tube to the middle of the Sample Pad, immediately apply one drop of Chase Buffer to the Sample Pad.



Adding Sample Type 3

Serum / Plasma
Add 50 µL of serum or plasma (precision pipette) to the Sample Pad.



4 Read Results

Read the Test Result between 15 and 30 minutes after the addition of the sample. **Do not read Test Results after 30 minutes.** The control line should appear for all results. If it does not appear, the results are invalid and should be repeated. Interpret any visible bar (even very faint) in the window as a valid result.

CONTROL

PATIENT

POSITIVE

NEGATIVE

INVALID

