NycoCard™ READER II

Abbott Diagnostics Technologies AS
Kjelsåsveien 161
P.O. Box 6863 Rodelokka
NO-0504 Oslo, Norway
www.abbott.com/poct

ISO 13485 certified company

© 2019 Abbott. All rights reserved. All trademarks referenced are trademarks of either the Abbott group of companies or their respective owners.
1116951 Rev. A 2019/02

REF 1116815

1116951 Rev. A 2019/02

EN	5
Intended use of NycoCard™ READER II	5
Contents of NycoCard™ READER II	5
Functional description	5
Basic theory	5
Installation and environment	5
Using the instrument	6
Operating procedure	6
Cleaning and maintenance	7
Disposal of the NycoCard™ READER II	7
Conformity to the European IVD directive and RoHS 2 directive	7
Troubleshooting	8
Symbols and abbreviations	9
Warranty	9
Technical specifications	10

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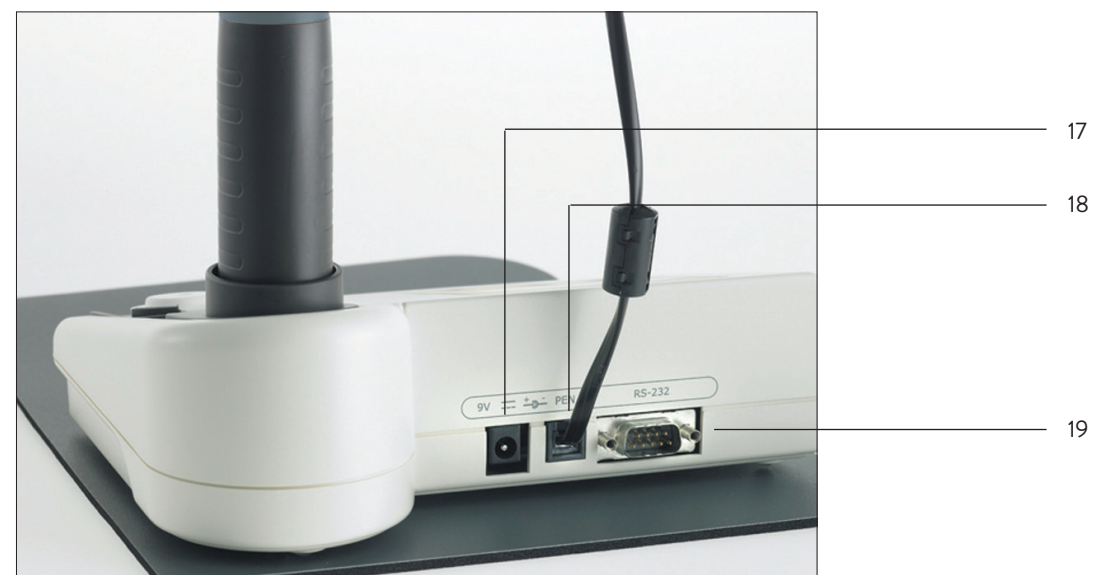
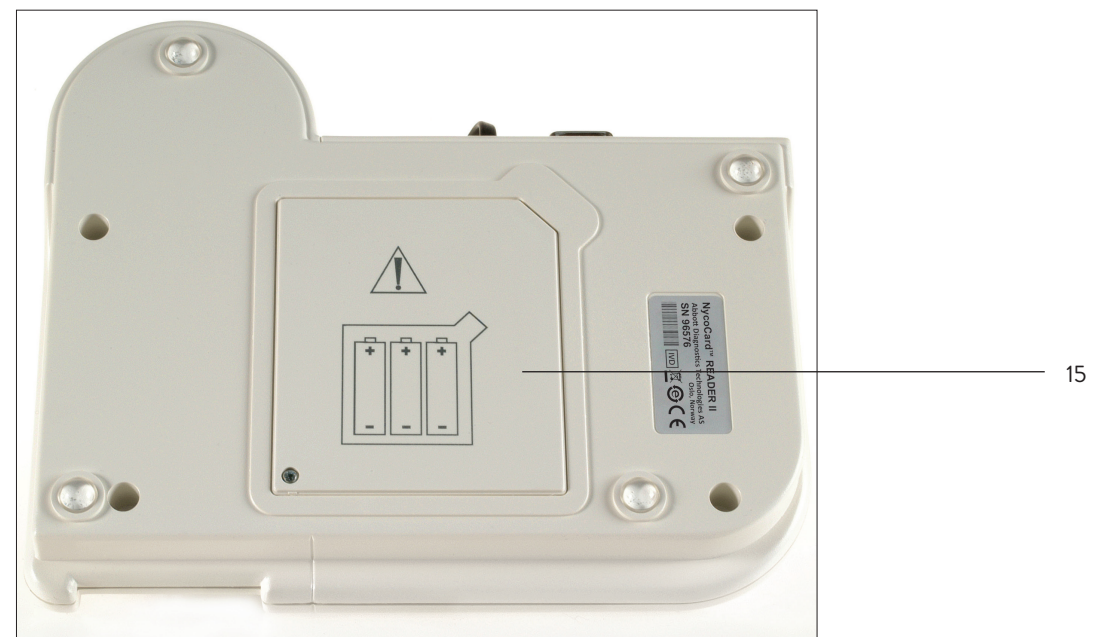
For professional near-patient testing and laboratory use.

Intended use of NycoCard™ READER II

The NycoCard™ Reader II is a small battery-powered reflectometer, designed to measure the coloured surfaces on the NycoCard Tests and calculate the concentration values. The coloured responses are measured by spectral reflectance in three parts of the visible spectrum.

 The NycoCard Reader II is calibrated by the manufacturer for the NycoCard Tests. Accurate results can only be assured when the NycoCard Reader II is used with the NycoCard Tests.

Contents of NycoCard™ READER II

The NycoCard Reader II Kit consists of:

NycoCard Reader II	1 unit
Battery charger	1 unit
Batteries (rechargeable)	3 units
A1 Calibration devices	10 units
A2 Pen tips	10 units
A3 Pen rings	10 units
Pad	1 unit
Instruction manual	1 unit

When unpacking, examine the contents of the package for any sign of shipping damage and check if the package is complete. Report the missing items or shipping damage to your supplier.

Functional description

The NycoCard Reader II consists of the instrument box (1), which is the operational and calculating unit, and the Reader pen (5), which is the measurement unit. The main components of the instrument box are the microprocessor, the LCD display (3) and the push buttons (2) for operation. The key elements of the Reader pen are the Light Emitting Diodes (LEDs) (11), a photodiode (12) and a light/dark detector (10). A pen cable (4) connects the box and pen together. The penholder (8) is located to the right of the display on the instrument box. In front of this holder, there is a lid (6), and on the rear side of the lid a white calibration device (9) is placed.

-  Do not interchange Reader pens between different instruments. The specific calibration data is stored in the box.
- The pen should be placed in the penholder when not in use. This protects the optics from dust and damages.

Basic theory

The Reader pen contains three pairs of LEDs: red, green and blue. Reflected light from the test sample is measured, relative to a white sample, by a light-sensitive photodiode circuit. When a specific test from the NycoCard Reader II Test Menu is chosen, the required LEDs illuminate the sample to be measured. The sample will reflect some of the light, which will be detected by the photodiode.

When the pen is placed over the sample and the pen sleeve is pushed down, the light/dark detector registers that the environment has changed from light to dark and the measurement is automatically started. The sleeve will ensure that the pen is held stable in a vertical position, and that external light is excluded from the sample.

Adjusting

This procedure compensates for the contribution of reflected light from the inside of the pen tip (14) and slight electrical current leakage in the electronic circuit. Adjusting is performed when the pen is placed in the penholder, i.e. in a dark chamber without any sample present.

-  It is important to adjust the instrument each time the instrument is turned on and when the pen tip is replaced.

White calibration

The White calibration is essential in reflectance measurements and adjusts for any differences in measuring conditions and variation of pen tips. The white calibration is performed on a special calibration device, which is stored under the lid, protected from light and dust.

-  The White calibration should be performed prior to each single, sporadic measurement and prior to each series of measurements (max 20 measurements per series).

Installation and environment

Recommended ambient temperature is 18 – 25°C and humidity must not be condensing. Ensure that the instrument is protected from dirt and direct sunlight.

-  Placement of the NycoCard Reader II should allow easy disconnection from mains A/C power at any time.
- Make sure that the instrument is acclimatised to room temperature before use.
- Water or any other liquid should not be in contact with the instrument.
- Please check the surroundings environment with respect to strong electromagnetic radiation. It might interfere with proper operation.

Your NycoCard Reader II is supplied with 3 rechargeable NiMH batteries (1.2 V) (16). Before using the new instrument, be sure that the batteries are sufficiently charged. The connection plug of the battery charger is inserted into the outlet (17) located at the rear side of the instrument box.

- Be sure that the batteries are inserted in the right direction.
- Be certain that the outlet voltage has the same rating as the charger.

The Reader pen has a modular plug (18). The NycoCard Reader II is equipped with RS-232 port for PC-connection (19).

Using the instrument

Functions

The instrument is operated by three push buttons:

To switch on the instrument and to screen the main menus and the sub menus.

To enter and activate the menu selected by the Select button.

To exit the current menu or function.

The menu

Four main menus are available for selection:

Main menu/Calibrate instructs the instrument to measure the White Calibration Device.

Main menu/Enter test contains sub menus corresponding to each of the NycoCard Tests that are available.

Main menu/Options contains three sub menus:

- *Enter language*: To change the language on the display.
- *HbA1c*: To select between the measuring units %, mmol/mol or % & mmol/mol.
- *New white*: To measure a new value of white when a message "Error Recalibrate" appears on the display. The white value will change with ageing, even if pen tips and calibration devices are replaced.

Main menu/Off followed by "ENTER" will turn off the instrument. Pressing QUIT and ENTER simultaneously will also turn off the instrument, regardless of the position in the menu.

Operating procedure

The following three steps must be performed before the result of a NycoCard Test is obtained:

1. Adjusting

Make sure that the pen is placed in the penholder and turn the Reader on. The instrument performs automatically "Adjusting". Allow the Adjusting procedure to complete without touching the pen. A single beep will indicate that the Adjusting is completed correctly.

- Make sure that the pen tip is free from dust and scratches.
- Make sure that the penholder is clean.

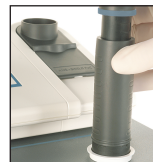
2. White calibration

Press ENTER to activate the menu and follow the instructions in the display. When the message "Place pen..." is displayed, lift the pen out of the holder and place it on the White Calibration Device. Hold the pen steady until a single beep indicates that the calibration is finished.

- Make sure that the white calibration surface is free from dust and scratches.
- Remember to close the lid after white calibration is performed.

3. Measuring the NycoCard Test

The requested NycoCard Test is chosen from Main menu / Enter test. While the pen is in the pen holder, press Enter shortly and follow the instructions on the display. The measurement starts when the pen sleeve is pushed down. After a few seconds the test result will be displayed as a concentration value.

- The NycoCard Test should be placed on the Reader pad (7) when measuring the result. Measuring without using the pad may lead to wrong results.
- Follow general laboratory precautions when handling blood samples. Be aware that the pen tip and pen ring will be in contact with diluted blood/urine. Wear gloves.
- Be aware that measurement is in progress when "Active Wait..." is shown on the display. Make sure that this message is not displayed before the sleeve is pushed down on the sample. Keep the sleeve down and steady until the result is displayed.

If the instrument is placed on stand-by for about 10 minutes, it will automatically switch-off. When the instrument is turned on again, the last measured value will automatically be displayed.

Cleaning and maintenance

Cleaning the instrument

Cleaning the exterior of the instrument should be performed whenever necessary. Most spills and stains can be removed with water or a mild detergent.

- Turn off the instrument and disconnect it from mains A/C power.
- Wipe the outside of the instrument with a clean, lint-free and non-abrasive cloth dampened in water or a mild detergent.
- Do not use a cloth for cleaning of the Reader pen tip or pen ring. See section Replacement of disposable parts.
- To disinfect the exterior of the analyzer, use a 1:10 solution of household bleach (i.e., 0.5 % sodium hypochlorite) or 70 % alcohol solution. The surface of the instrument should be exposed to the disinfectant for at least 10 minutes.
- Allow the instrument to air dry.

Replacement of disposable parts

The NycoCard Reader II has three replaceable accessories, the white calibration device, (9) the pen tip (14) and the pen ring (13). It is recommended that these accessories are inspected regularly and replaced if dirty or damaged.

Replacing the pen tip: Wear gloves. Carefully snap off the used pen tip with your fingers. Replace with a new pen tip.

Replace the pen ring: Wear gloves. Hold the case shield with one hand and carefully wriggle off the pen ring with the other hand. Replace with a new pen ring. Make sure it fits correctly.

Replacing the calibration device: Remove the device by simultaneously pressing the small handle down and the opposite side upwards. Insert the new calibration device.

- Be careful not to touch the optics of the Reader pen.
- Any dirt or damages of the pen tip or white calibration device may cause wrong measurements.

Replacing the batteries:

1. Use a screwdriver to open the battery lid (15).
2. Remove the old batteries (16).
3. Replace the batteries with three new batteries. Be sure that the batteries are inserted in the right direction.
4. Replace the battery lid. New batteries need to be charged before using the instrument.

- When disposing of the batteries, please follow recommendations from the supplier.

Disposal of the NycoCard™ READER II

The NycoCard Reader II may be contaminated by infectious agents from biological material during testing. Used devices therefore carry a risk of transmission of infection. When disposing of a used NycoCard Reader II or potential contaminated accessory, please do so in accordance with the applicable regulations in your country.

For correct disposal according to the Directive 2012/19/EU (WEEE), please contact your local NycoCard Reader II supplier. You may also contact your local council or authority for information about correct disposal of your device.

Conformity to the European IVD directive and RoHS 2 directive

The NycoCard Reader II meets all provisions in the Directive 98/79/EC on *in vitro* diagnostic (IVD) medical devices and in the Directive 2011/65/EU on the restriction of the use of certain hazardous substances in electrical and electronic equipment (RoHS 2).

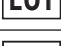
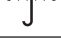
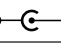
Troubleshooting

The following conditions or problems may occur when using the NycoCard Reader II. If your problem still exists after corrective action is taken, please contact your local distributor.

Problem	Possible cause	Corrective action
"Adjust ERROR".	<ol style="list-style-type: none"> 1. Pen tip is dirty or damaged. 2. Intermittent temperature changes or condensing conditions. 	<ol style="list-style-type: none"> 1. Replace Pen tip, restart the instrument, perform Adjusting and White calibration. 2. Perform New White in Main menu Option, restart the instrument, perform Adjusting and White calibration. <p>If "Adjust Error" still appears, please contact your local distributor.</p>
"Battery low" appears on the display.	<ol style="list-style-type: none"> 1. Batteries discharged. 2. Batteries improperly inserted into the battery holders. 3. One of the batteries has short-circuited and is not recharging. 	<ol style="list-style-type: none"> 1. Recharge the batteries. 2. Reinsert batteries. 3. Replace all batteries.
Display screen is blank or unreadable.	<ol style="list-style-type: none"> 1. Batteries discharged. (see other causes under "Battery low"). 2. Sub-optimal viewing conditions Light too dim. 	<ol style="list-style-type: none"> 1. Recharge the batteries. Check polarity and that the batteries are of the correct type. 2. Increase light level or change light position.
Display fades out during "Adjusting".	<ol style="list-style-type: none"> 1. Batteries discharged. 	<ol style="list-style-type: none"> 1. Recharge batteries.
"Calibrate white/ ERROR! Recalibr." appears on the display.	<ol style="list-style-type: none"> 1. Calibration device used is dirty, scratched or not totally white 2. Ageing of the instrument has changed the internal white value. 3. Pen tip is dirty or damaged. 	<ol style="list-style-type: none"> 1. Replace calibration device and recalibrate. 2. Measurement of new white value. 3. Replace the pen tip. Recalibrate.
"Calibrate white Active Wait..." persists on the display screen.	<ol style="list-style-type: none"> 1. Pen cable not properly inserted. 	<ol style="list-style-type: none"> 1. Check that the pen cable connector plug is properly inserted into the receptacle of the instrument. Recalibrate.
Program will not proceed from "Place pen" to "Active. Wait..."	<ol style="list-style-type: none"> 1. Room light too strong or unit in direct sunlight. 	<ol style="list-style-type: none"> 1. Reduce light/avoid sunlight on device. Perform adjusting and white calibration.
"RAM ERROR" / "Test data error"		<ol style="list-style-type: none"> 1. Contact your local distributor.

Symbols and abbreviations

The following symbols and abbreviations are used in the labelling and instructions for the NycoCard Reader II.

Symbols/Abbreviations	Explanation
	Conformity to the European directive 98/79/EC on <i>in vitro</i> diagnostic medical devices
	Conformity to the RoHS 2 directive
	<i>In Vitro</i> diagnostic medical device
	Catalogue number
	Lot number
	Serial number
	Manufacturer
	Date of manufacture (year-month-day)
	Expiry date (year-month)
	Temperature limitation
	Keep dry
	Consult instructions for use
	Caution
	General symbol for recovery / recyclable
	Biological risk
	Conformity to European Directive 2012/19/EU Waste Electric and Electronic Equipment (WEEE)
	Direct current
	Positioning of cell
	Protective isolation protection class II IEC 417
	Polarity, pos (+) in the centre
LED	Light Emitting diode
LCD	Liquid Crystal Display

Warranty

Abbott Diagnostics Technologies AS guarantees a warranty period of 12 months from the date of delivery to the user. For this period it is guaranteed that the NycoCard Reader II works within quality requirements valid on the date of manufacture release.

Abbott Diagnostics Technologies AS is prepared to compensate for faulty material or malfunctions within the limitations of the warranty.

The warrant is limited to providing a replacement NycoCard Reader II.

Tampering with the internal components, damage due to operating errors, misuse and overlooking essential information with respect to warnings and precautions described in this User Manual will invalidate the warranty.

The acknowledgment of claims shall immediately be reported to your local NycoCard Reader II supplier.

Technical specifications

Instrument

Size, instrument box:	200x170x70 mm
Size, Reader pen:	L: 144 mm, D: 29.5 mm
Weight:	540 g (incl. Reader pen and batteries)
Display type:	LCD, 2x16 characters
Computer:	Intel 80 C32
Memory capacity:	Programme memory capacity: 64 Kbytes
Test storage capacity:	2 Kbytes
Power supply:	Batteries, battery charger
Safety regulations:	IEC61010-1, IEC 61010-2-101, IEC 61326-2-6

Measuring conditions

Temperature range:	15-35°C ¹⁾ (The recommended range is 18 - 25°C)
Relative humidity:	0-90%

Avoid direct sunlight or too dim light

¹⁾ See special instructions for each NycoCard Test

Storage and transportation conditions:

Temperature:	- 40 - 70°C
Relative Humidity:	10 - 93 % at 40°C

Batteries

Rechargeable NiMH batteries only, size AA,	Size AA, 1.2 V
--	----------------

Battery charger

Mascot type 9725	
Input:	100-240 VAC
Output:	9V DC/6 W
Dimensions without plug (LxWxH):	75x32x40 mm
Weight:	65 g

Plug

3630+ (coaxial type)	
Polarity:	pos. (+) in centre
Length:	12.0 mm
Inner diameter:	2.1 mm
Outer diameter:	5.5 mm

Ordering information

Product	Product no.	Quantity
NycoCard Reader II	1116815	1 unit
A1 Calibration devices	1116821	1x10 units
A2 Pen tips	1116823	1x10 units
A3 Pen rings	1116822	1x10 units

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